Digital Still Recorder

Instructions for Use page 2(E)

Before operating the unit, please read this manual thoroughly and retain it for future reference.

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DKR-700/700P

Owner's Record

The model and serial numbers are located at the rear.
Record the serial number in the space provided below.
Refer to these numbers whenever you call upon your Sony dealer regarding this product.

Model No. DKR-700 Serial No. _____

WARNING

To prevent fire or shock hazard, do not expose the unit to rain or moisture.

To avoid electrical shock, do not open the cabinet. Refer servicing to qualified personnel only.

This symbol indicates type B equipment classified in accordance with IEC Publication 601-1 Safety of medical electrical equipment. (DKR-700 only)

This symbol indicates equipotential terminal which brings the various parts of a system to the same potential. (DKR-700P only)

For the customers in the USA

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

You are cautioned that any changes or modifications not expressly approved in this manual could void your authority to operate this equipment.

This device requires shielded interface cables to comply with FCC emission limits.

CAUTION


The use of optical instruments with this product will increase eye hazard.

CAUTION


Use of this product other than directed may result in injury.

For DKR-700P

The following label is located on the top of the unit.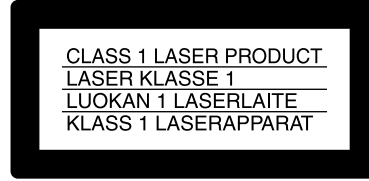

The following label is located inside the unit.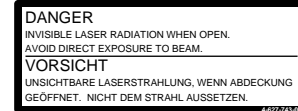


LITHIUM BATTERY

Should only be changed by technical personnel.
There is a risk of explosion if handled improperly.

Important safeguards/notices for use in the medical environments

1. All the equipments connected to this unit shall be certified according to Standard IEC601-1, IEC950, IEC65 or other IEC/ISO Standards applicable to the equipments.
2. When this unit is used together with other equipment in the patient area*, the equipment shall be either powered by an isolation transformer or connected via an additional protective earth terminal to system ground unless it is certified according to Standard IEC601-1.

* Patient Area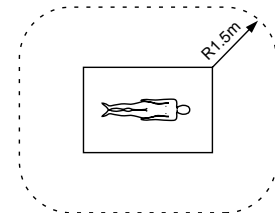

3. The leakage current could increase when connected to other equipment.
4. This unit generates, uses, and can radiate frequency energy. If it is not installed and used in accordance with the instruction manual, it may cause interference to other equipment. If this unit causes interference (which can be determined by unplugging the power cord from the unit), try these measures: Relocate the unit with respect to the susceptible equipment. Plug this unit and the susceptible equipment into different branch circuits. Consult your dealer.

WARNING

When installing the unit, incorporate a readily accessible disconnect device in the fixed wiring.

If a fault should occur during operation of the unit, operate the disconnect device to switch the power supply off.

WARNING!

The DKR-700 is intended to be used for recording video images obtained from video cameras and medical imaging

systems, primarily for the purposes of image archival and teaching/education. When any of the compressed modes (FINE, NORMAL, QUICK2, QUICK1) are utilized, images stored may not necessarily retain the full resolution and image information as delivered by the originating source modality.

This is because JPEG lossy compression is employed, and, by definition, this type of compression discards certain information which cannot ever be retrieved.

Table of Contents

Overview	4(E)	Searching for an Image Using the Image	
Features	4(E)	Index	29(E)
Usage Example	5(E)	Searching for an Image Using Information	
Usage Precautions	7(E)	Search List	29(E)
Location and Function of Parts	8(E)	Image Data Management	
DKR-700/700P Main Unit	8(E)	(Using the RM-C700)	32(E)
RM-C700 Remote Control Unit (Option)	14(E)	Keys Used to Input ID Numbers, Image	
Preparation	16(E)	Names and Disc Names	32(E)
Connections	16(E)	Setting/Changing the Disc Name	33(E)
Setting the DIP Switch	17(E)	Setting/Changing an ID Number	33(E)
MiniDiscs (MD Data Discs)	17(E)	Setting/Changing an Image Name	35(E)
Activating the DKR-700/700P and		Deleting Images	37(E)
Inserting/ Removing Discs	18(E)	Erasing Images One at a Time	37(E)
Initializing Discs	19(E)	Erasing All Images on the Disc	37(E)
Setting the Clock	19(E)	Menu Operations	38(E)
Checking the Setup/Operating Condition of		Menu Structure	38(E)
the DKR-700/700P	19(E)	Buttons/Keys for Menu Operations	39(E)
Recording Images	22(E)	Structure and Usage of the Main Menu	39(E)
Preparing to Record	22(E)	Structure and Usage of the Source Setup	
Recording One Image at a Time — Step		Menu	40(E)
Recording	23(E)	Structure and Usage of the Capture/Rec Setup	
Recording Images Automatically — Interval		Menu	41(E)
Recording	24(E)	Structure and Usage of the Play Setup	
Recording with a 1-Contact Foot Switch	24(E)	Menu	43(E)
Recording with the Optional FS-30 Foot		Structure and Usage of the Disp Setup	
Switch	25(E)	Menu	44(E)
Playing or Searching for Images	26(E)	Structure and Usage of the System Setup	
Preparing to Playback or Search	26(E)	Menu	45(E)
Playing One Image at a Time — Step		Structure and Usage of the User Setup	
Playback	27(E)	Menu	47(E)
Playing Images Automatically — Interval		Appendixes	49(E)
Playback	28(E)	Specifications	49(E)
Playing an Image by Its Image Number		Error Messages and Warning Messages	51(E)
— Direct Access Playback		Troubleshooting	53(E)
(Using the RM-C700)	28(E)	Timing for Capturing an Image Using a Foot	
		Switch	53(E)

Overview

The DKR-700/700P Digital Still Recorder is a device for recording still image to a MiniDisc (MD Data Disc) and filing it for later playback.

Features

MiniDisc as a recording medium

The DKR-700/700P records images to a data MiniDisc (MMD-140 MD Data Disc or equivalent: 140 Mbytes) complying with the Sony picture MD format. This magneto-optical recording medium not only boasts a high degree of permanence and reliability, but also reduces still image storage cost by recording up to 1000 images on a single disc.

Recording with digital compression

Using the latest digital technology, the DKR-700/700P provides image quality better than 480 lines of horizontal resolution. The DKR-700/700P employs widely-used JPEG compression method and TIFF format for uncompressed images, making it possible to examine images using commonly available software.

Selectable image quality

When recording, one of five image quality modes (uncompressed, fine, normal, quick 2, and quick 1) can be selected. This allows the DKR-700/700P to be used in a wide variety of applications, ranging from the medical field, which requires high image quality, to filing systems which emphasize the storage of large numbers of images and must be quickly searchable.

Simultaneous recording of image information

When recording an image, the date (year, month, day, hour, minute, and second) and an image number are automatically recorded or updated. With the connection of the optional RM-C700 Remote Control Unit or a computer, it becomes possible to attach an image ID number (up to 15 digits) and an image name (up to 15 alphanumeric characters).

Rich search functionality

The DKR-700/700P allows a desired image to be found quickly. Searches may be performed on image information such as the image number, ID number, image name, or date, and an image may also be located via image indexes (25 thumbnail images per index).

Setup using menus

The menus allow basic selections such as video input/output connectors and image recording quality, as well as the setup of recording and playback modes, status/setup information display, and system control.

Various video input/output connectors

The DKR-700/700P is equipped with RGB, composite, and S video input/output connectors. A wide variety of input sources may be selected, from common sources such as video cameras and VCRs to LaserDisc players and medical imaging equipment. The DKR-700/700P may be connected to output devices such as video monitors, video projectors, and video printers.

External control functionality

The DKR-700/700P is equipped with an RS-232C interface connector and a SCSI interface connector. Besides the optional RM-C700 Remote Control Unit or a computer, an optional 3-contact foot switch FS-30 or a 1-contact foot switch may be connected, allowing external control of the DKR-700/700P.

Record and playback modes selection according to purpose

Images may be recorded and played back one at a time (step recording/playback), or may be recorded or played back in order (interval recording/playback).

Synchronization with imaging equipment

Since the DKR-700/700P can generate a flash trigger signal when an image is recorded, images may be acquired in synchronization with a flash. When paired with the Sony DXC-950 series (DXC-970MD for


medical use) Video Camera, using the slow electronic shutter feature enables clear, sharp recording even of low-illumination images such as those from fluorescence microscopes.

Usage Example

Image filing system for medical or industrial measurement applications

The DKR-700/700P may be coupled with medical instruments or with factory measurement equipment to form an image filing system. By using the supplied DKR-700 Plug-In Software (running under Microsoft

Windows¹⁾ or MacOS²⁾), image data can be transferred to a computer connected to the DKR-700/700P via a SCSI interface.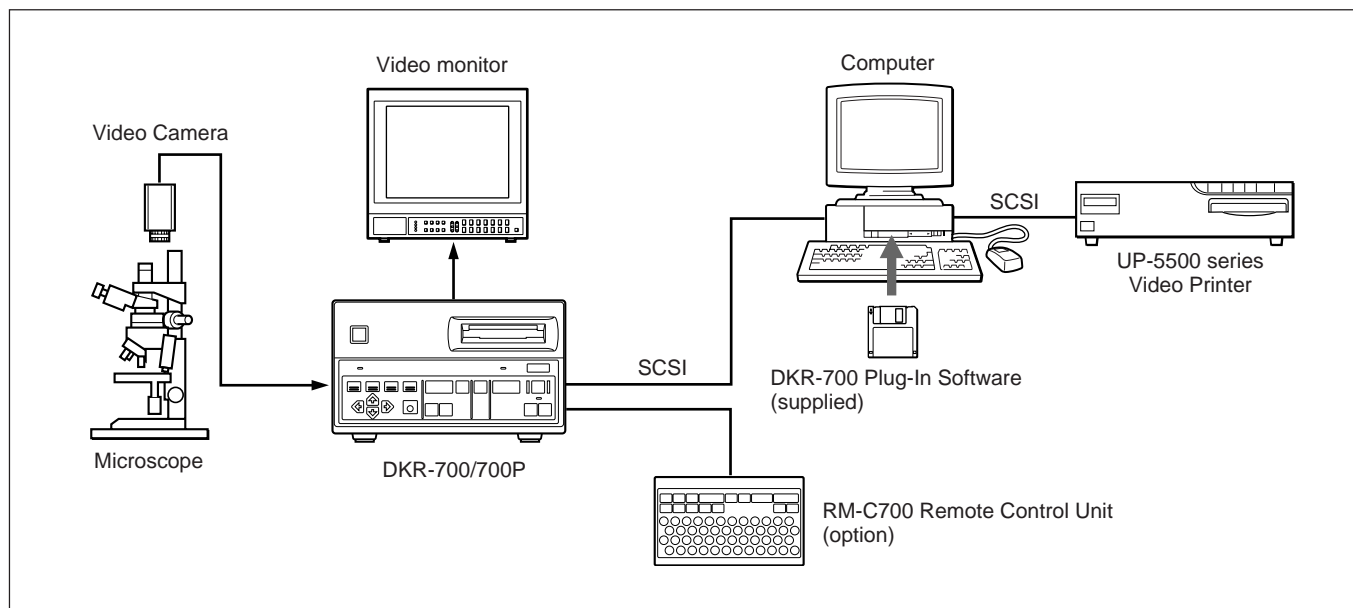

- 1) Windows is a trademark of Microsoft Corporation.
- 2) On the PowerBook, the Plug-In Software does not function properly.

Overview

Computer-based image processing system

By using the supplied DKR-700 Plug-In Software (running under Microsoft Windows or MacOS¹⁾), image data may be exchanged with a computer

connected to the DKR-700/700P via a SCSI interface. Images can be transferred from the DKR-700/700P to a computer for processing and use in CG still production.


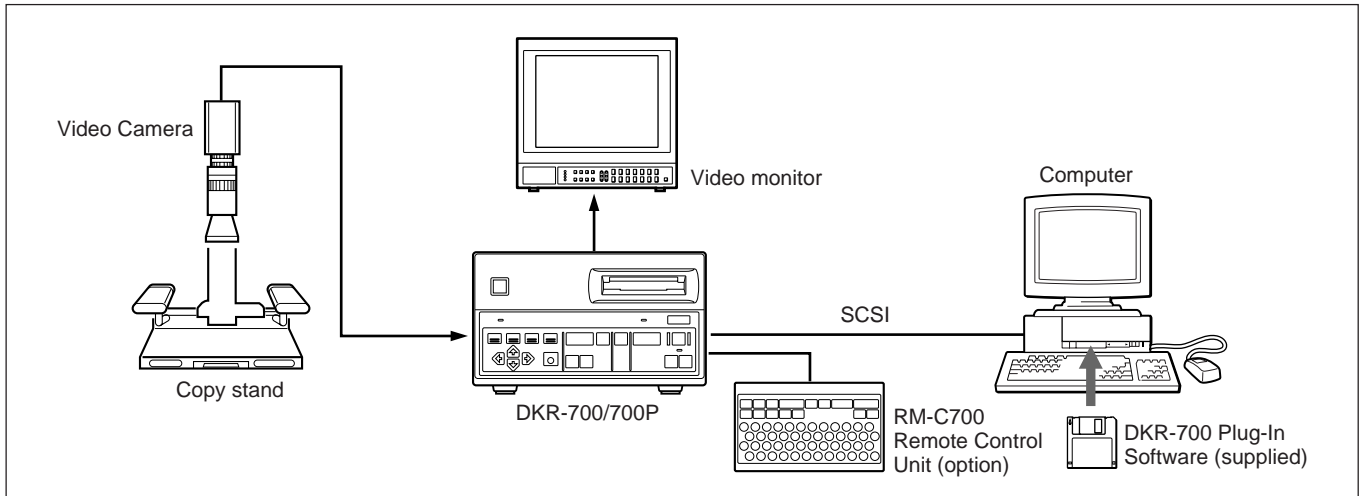
Photo proofing system

When taking photographs with a film camera, the same images may be captured simultaneously using the DXC-950 series (DXC-970MD for medical use) Video Camera and stored with the DKR-700/700P to allow proofing of the images before the photographs have been developed, printed, or enlarged. The image can

be captured via automatic recording, based on the timing of a trigger signal from the video camera, or the timing of the capture can be controlled by a flash signal generated by the DKR-700/700P. Besides proofing in a photo studio, the same system can also be used in industrial or medical inspection photography.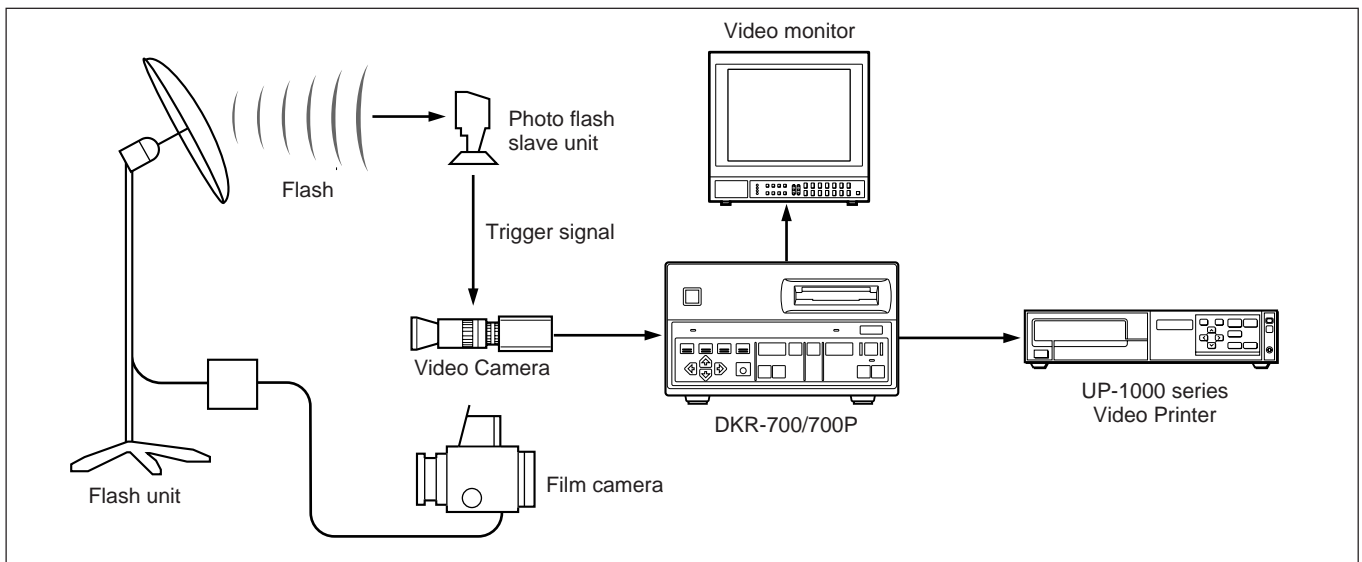

1) On the PowerBook, the Plug-In Software does not function properly.

Usage Precautions

Operation and storage locations

Avoid operation or storage in any of the following places.

- Location subject to extremes of temperature (operating temperature range 5°C to 35°C (41°F to 95°F))
- Location subject to direct sunlight for long periods, or close to heating appliances (Note that the interior of a car left in summer with the windows closed can exceed 50°C (122°F).)
- Damp or dusty places
- Location subject to severe vibrations
- Location near equipment generating strong electromagnetic emissions
- Location near transmitting stations generating strong radio waves

Operate the unit in a horizontal position

This unit is designed to be operated in a horizontal position. Do not operate it on its side, or tilted through an excessive angle (exceeding 20°).

Avoid violent impacts

Dropping the unit, or otherwise imparting a violent shock to it, is likely to cause it to malfunction.

Do not obstruct ventilation openings

To prevent the unit from overheating, do not obstruct ventilation openings, by for example wrapping the unit in a cloth while it is in operation.

Regular check

There is no need for regular checks.

Cleaning

If the casing or panel is dirty, wipe it gently with a soft dry cloth. In the event of extreme dirt, use a cloth steeped in a neutral detergent to remove the dirt, then wipe with a dry cloth. Applying alcohol, thinners, insecticides, or other volatile solvents may result in deforming the casing or damaging the finish.


Shipping

- Always remove the disc before shipping the unit.
- Pack the unit in its original carton or equivalent packing, and take care not to impart violent shocks in transit.

Location and Function of Parts

DKR-700/700P Main Unit

Power supply and data recording/deletion operations (front panel)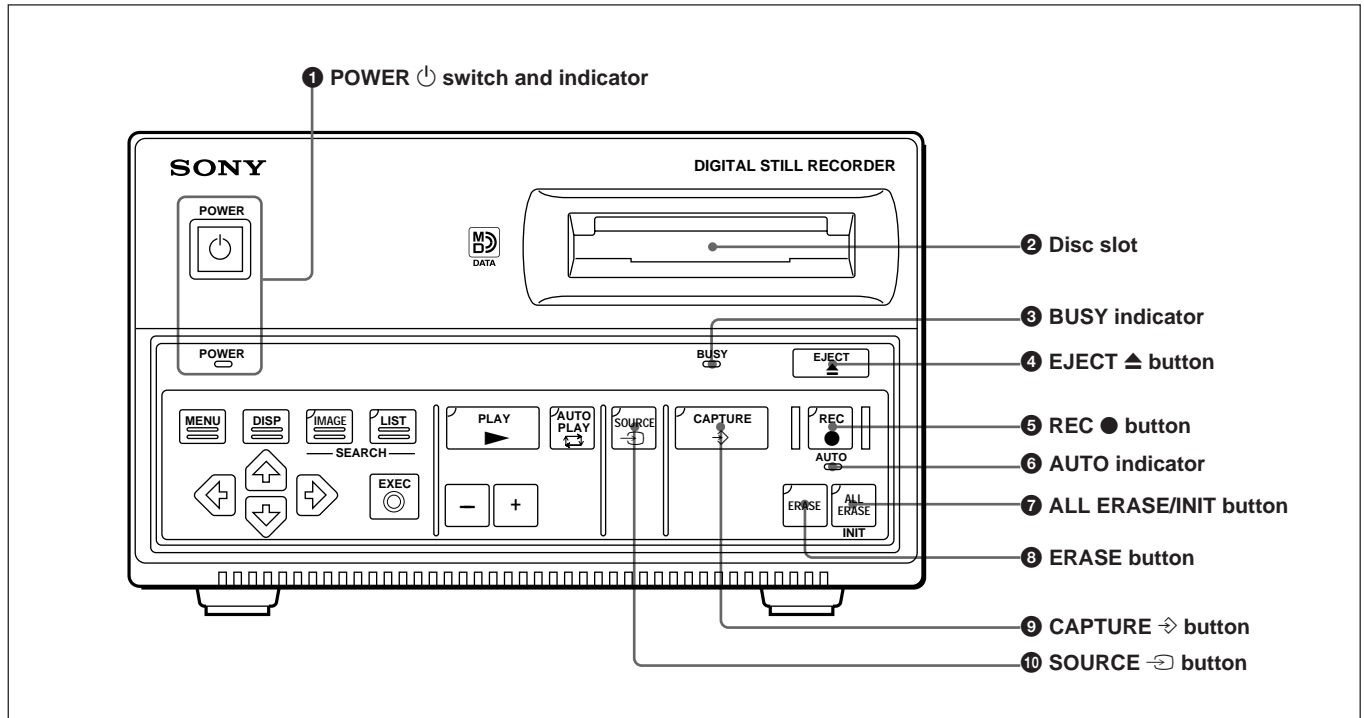

1 POWER switch and indicator

Pressing this switch once turns on power to the unit, and pressing it again turns the unit off. When the power is on, the indicator will light up. If the BUSY indicator is lit or blinking when you press the POWER switch to power off, the unit powers off after internal processing is completed.

2 Disc slot

For inserting discs.

3 BUSY indicator

When some operation is being performed on the disc, or when some data remains on the DKR-700/700P's memory, this indicator will be lit or will blink. When the indicator is lit, all buttons except the EJECT button will have no effect.

Caution

Do not disconnect the AC power cord from the outlet while the BUSY indicator is lit or blinking. If the power is disconnected while the disc is in operation, **data stored on the disc may be lost or damaged.**

4 EJECT button

Pressing this button will eject the disc.

5 REC (record) button

When the button is pressed, it will light, and recording will begin. When recording is complete, it will go out.

6 AUTO (automatic record) indicator

When an automatic record is being performed (using the menus), this indicator will light.

For more information on automatic recording, see "Recording Images Automatically — Interval Recording" on page 24(E).

7 ALL ERASE/INIT button

This button is used to erase all images recorded on a disc. It is also used to initialize discs. Insert a disc which is to be erased, and press the button. The button will begin to blink. When the button is pressed a second time while it is blinking, it will light steadily, and the all erasing will begin. When inserting a disc which is not initialized or initialized to a format incompatible with this unit, the button will begin to blink automatically. To initialize the disc press the button while it is blinking.

8 ERASE button

When this button is pressed while using the PLAY button to play back an image, the button will blink. When the button is pressed again, it will light and the image currently being played will be erased. When erasing is complete, it will go out.

9 CAPTURE ⇨ button


When this button is pressed, it will light, and the video signal being received will be captured in the DKR-

700/700P's memory. The video signal will simultaneously be output from the video output connector.

10 SOURCE ⇨ button

When this button is pressed it will light, and the video signal being received will be output from the video output connector.

Menu, image playback, and search (front panel)

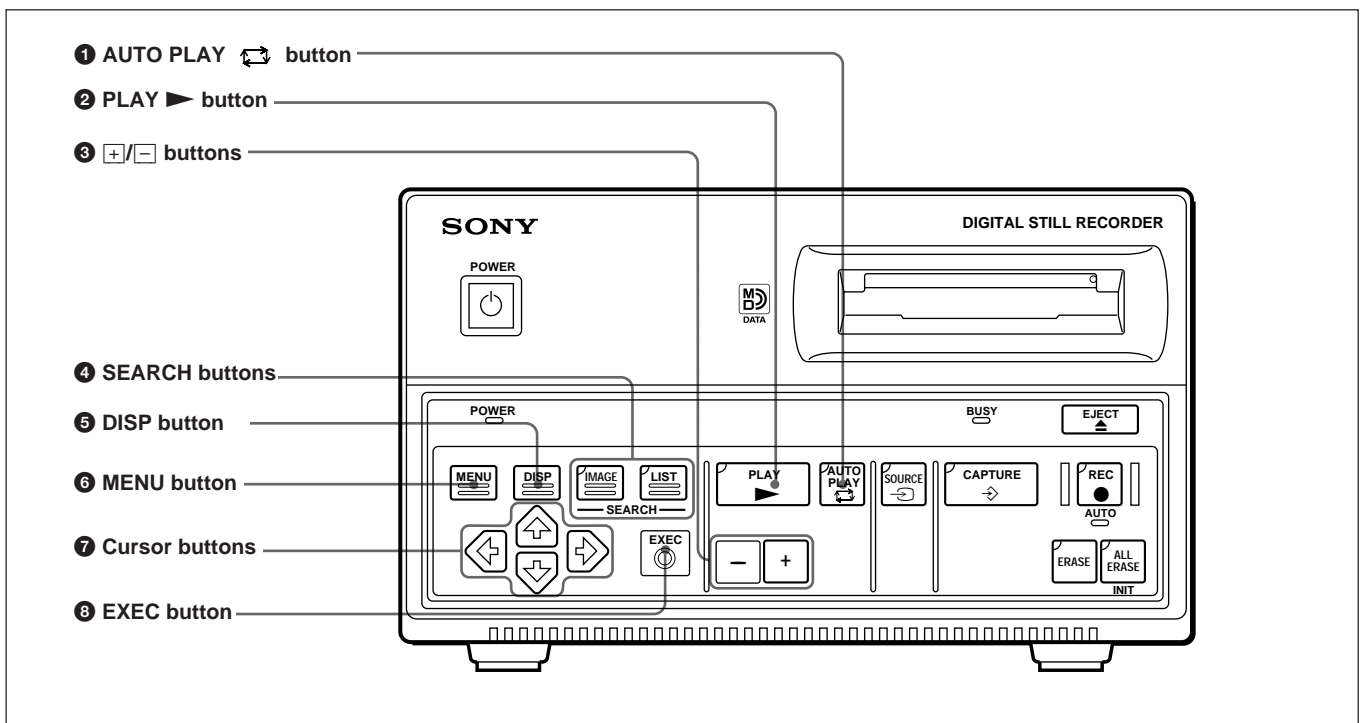1 AUTO PLAY ⇨ button

When this button is pressed it will light, and all images stored on disc will be played back in turn. The playback direction and the intervals between each image depend on parameters set in the menu.

2 PLAY ► button

When this button is pressed it will light, and one image stored on the disc will be played back. When this button is pressed after selecting a desired image using the search display (see *SEARCH buttons* 4), that image will be the one played. Either a main image or a “quick access” image may be played, according to what is selected in the menu.

3 +/− buttons

When the PLAY button 2 is lit, pressing the + or − button will cause a different image to be played back. The search images and search information lists are changed using the search display (see *SEARCH buttons* 4).

+ button: Each time this button is pressed, a image with higher image number will be displayed.

− button: Each time this button is pressed, a image with lower image number will be displayed.

Note

If there are images already recorded on the disc with a different type of device, the image numbers may in some cases not be consecutive.

Location and Function of Parts

④ SEARCH buttons

When one of the buttons is pressed, it will light, and a search display will appear on the screen. To search by image, press the IMAGE button. To search by image information, press the LIST button.

When the IMAGE button is pressed: Images stored on disc will be displayed as thumbnails (25 images on screen at once). When an image is selected using the cursor buttons ⑦ and the PLAY button ② is pressed, that image will be displayed on screen.

When the LIST button is pressed: Information about images stored on disc (image number, image name, ID number, date) will be displayed as a list. When an image is selected using the cursor buttons ⑦ and the PLAY button ② is pressed, that image will be displayed on screen.



⑤ DISP (display) button

When this button is pressed, the DKR-700/700P's setup and status information will be superimposed on the screen. The items to be displayed may be set using the menus. When the button is pressed again, the information display will disappear.

⑥ MENU button

When this button is pressed, the menu display will appear on the screen. When the button is pressed again, the menu display will disappear.

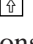
⑦ Cursor buttons

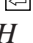In the menu display (*see MENU button ⑥*), these buttons are used to select menu items (the  and  buttons) and to change settings and conditions (the  and  buttons). In the search display (*see SEARCH buttons ④*), the cursor buttons are used to select a desired image.

⑧ EXEC (execute) button

This button is used to go from the main menu to a setup menu, and to confirm changes made to settings in the menu. Besides menu operation, this button is used to record the data on the DKR-700/700P's memory. (If some data remains on the memory, the BUSY indicator will blink.)

Power supply and video input/output connectors (rear panel)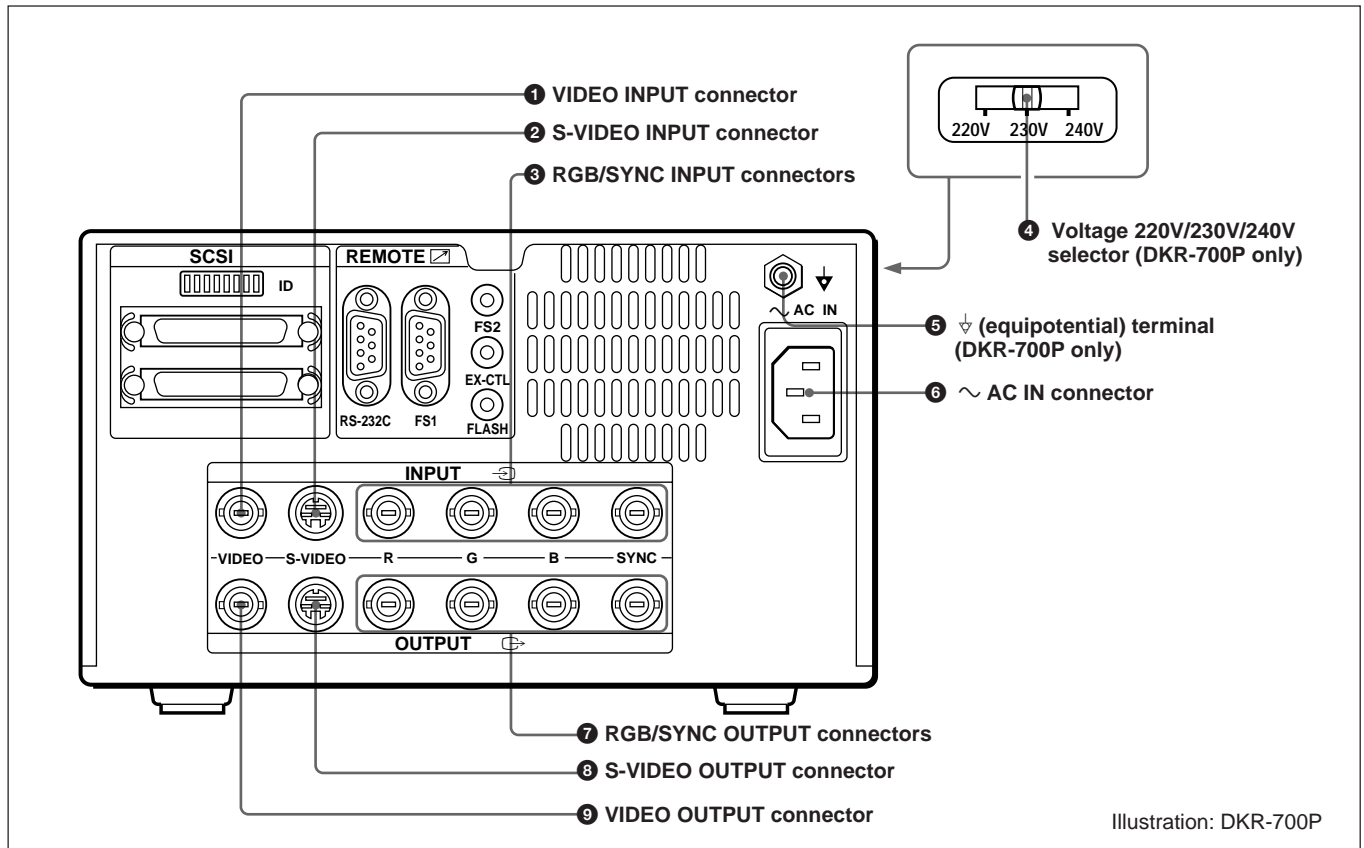

1 VIDEO INPUT connector (BNC-type)

Connect to the video output connector of a video camera, VCR, etc.

For the electrical specification of this connector, see page 49(E).

2 S-VIDEO INPUT connector (mini-DIN 4-pin)

Connect to the S video output connector of a video camera, VCR, etc.

For the electrical specification of this connector, see page 49(E).

3 RGB/SYNC INPUT connectors (BNC-type)

Connect to the RGB and SYNC output connectors of a video camera, etc.

For the electrical specification of these connectors, see page 49(E).

4 Voltage 220V/230V/240V selector (DKR-700P only)

Set the position which meets voltage in the mains line prior to the operation of the unit. Otherwise the unit may be damaged.

5 ⚡ (equipotential) terminal (DKR-700P only)

Connect to the system ground as required.

Location and Function of Parts

⑥ ~ AC IN (inlet) connector (3-pin)

Connect to the AC power source, using the supplied AC power cord.

⑦ RGB/SYNC OUTPUT connectors (BNC-type)

Connect to the RGB and SYNC input connectors of a video monitor, video printer, etc.

For the electrical specification of these connectors, see page 49(E).

⑧ S-VIDEO OUTPUT connector (mini-DIN 4-pin)

Connect to the S video input connector of a video monitor, etc.


For the electrical specification of this connector, see page 49(E).

⑨ VIDEO OUTPUT connector (BNC-type)

Connect to the video input connector of a video monitor, VCR, etc.

For the electrical specification of this connector, see page 49(E).

Remote connectors (rear panel)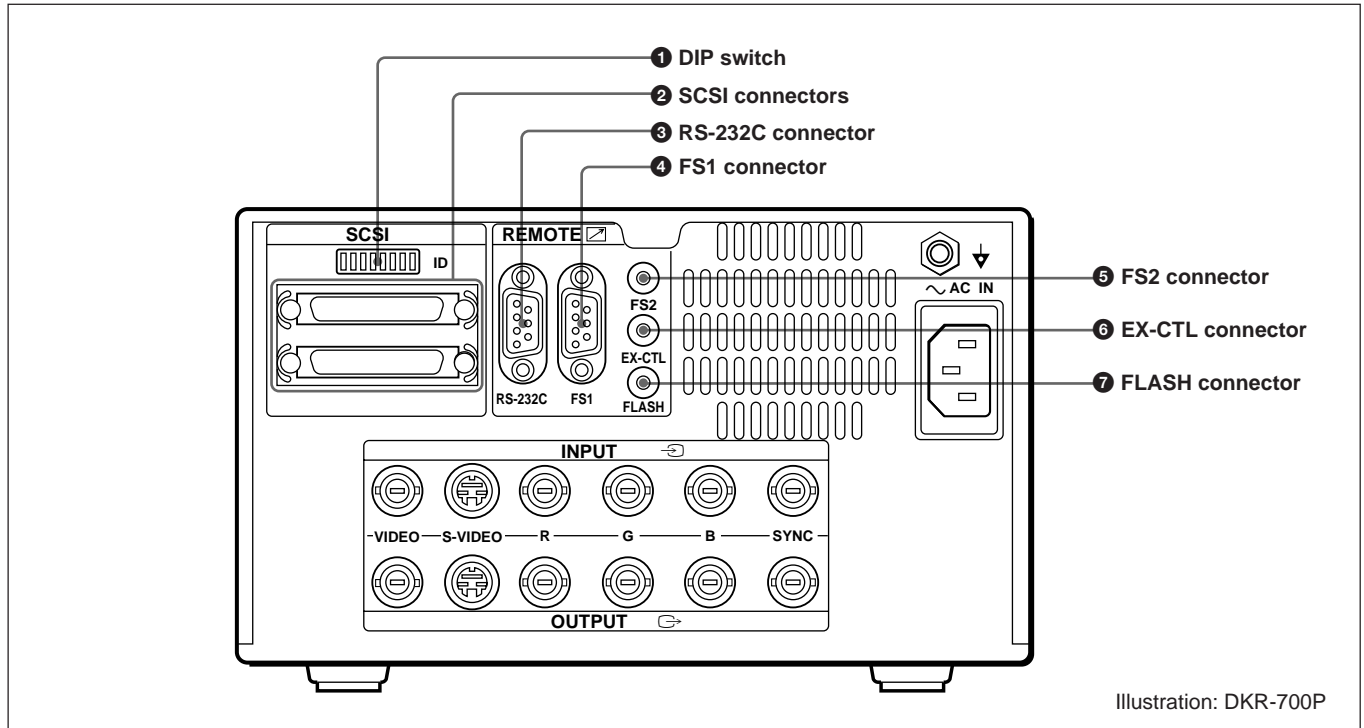

1 DIP switch

Used to set the SCSI-ID and operation mode.

2 SCSI connectors (50-pin, half-pitch)

Used to connect to a computer to exchange image data.

For the electrical specification of these connectors, see page 50(E).

3 RS-232C connector (D-sub 9-pin)

Used to provide external control of the DKR-700/700P, by connection to the optional RM-C700 Remote Control Unit or to a computer. Select which of the two is connected using the menus.

For the electrical specification of this connector, see page 49(E).

4 FS1 connector (D-sub 9-pin)

Used to connect an optional FS-30 Foot Switch.

5 FS2 connector (stereo mini-jack)

Used to connect a 1-contact foot switch.

For the electrical specification of this connector, see page 50(E).

6 EX-CTL (external control) connector (mini-jack)

Used to control external devices from the DKR-700/700P.

For the electrical specification of this connector, see page 50(E).

7 FLASH connector (mini-jack)

Used to connect to the external trigger connector of a flash unit.


For the electrical specification of this connector, see page 50(E).

Location and Function of Parts

RM-C700 Remote Control Unit (Option)

Record/play operations


For keys, see the descriptions of corresponding buttons on the DKR-700/700P.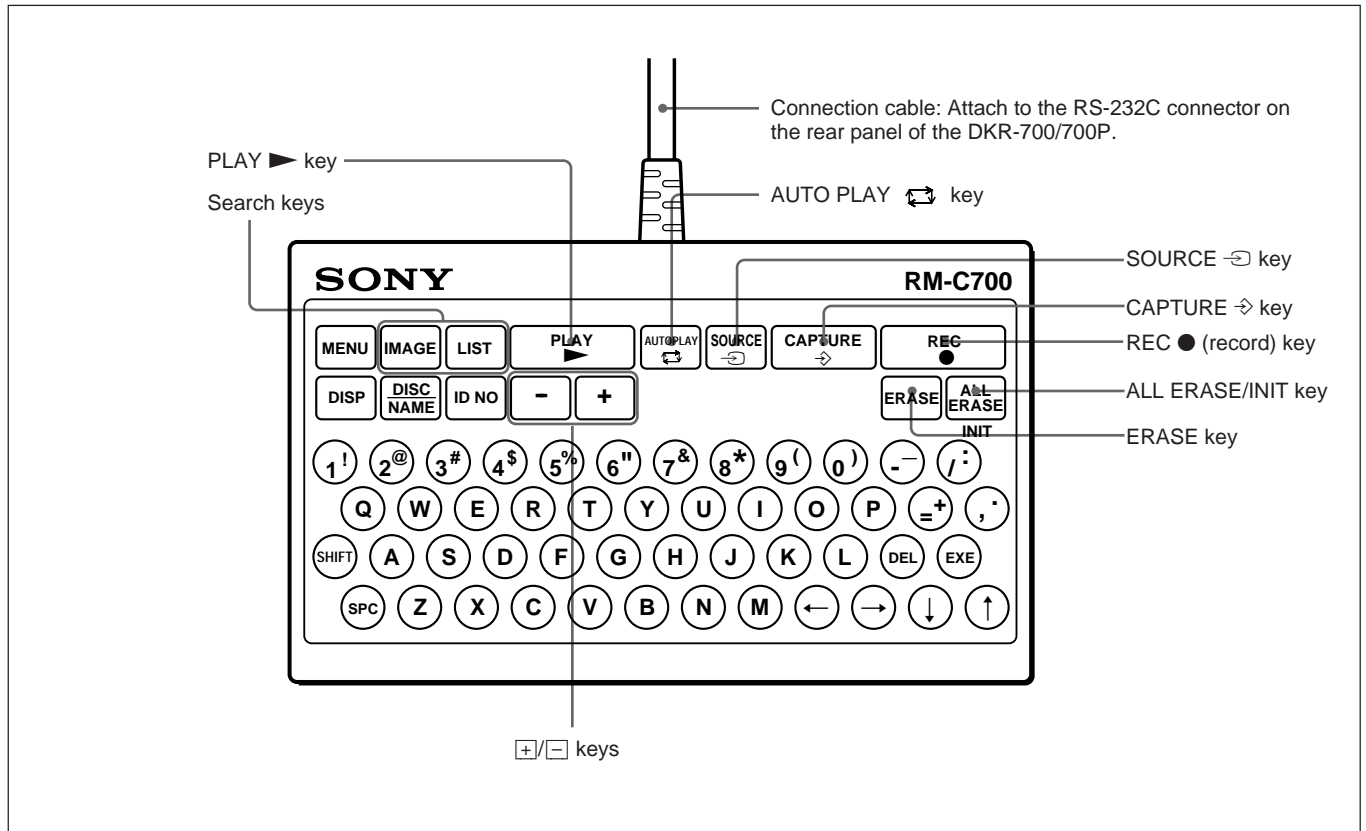

Menu/character entry operations

For the MENU, DISP, cursor, and EXE keys, see the descriptions of corresponding buttons on the DKR-700/700P.


For more information on character entry keys, see "Keys Used to Input ID Numbers, Image Names and Disc Names" on page 32(E).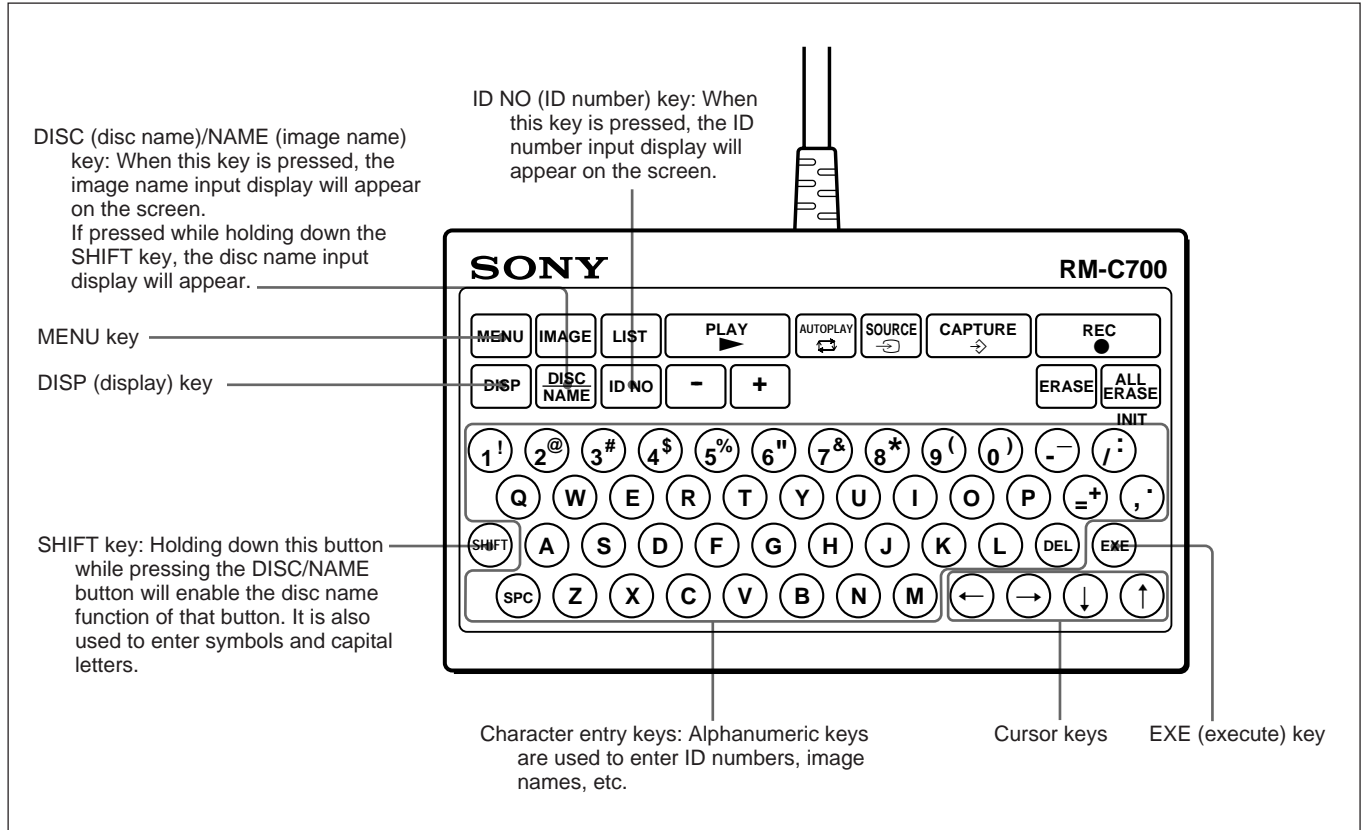

Preparation


Connections

Connect equipment required to the DKR-700/700P as shown in the figure below.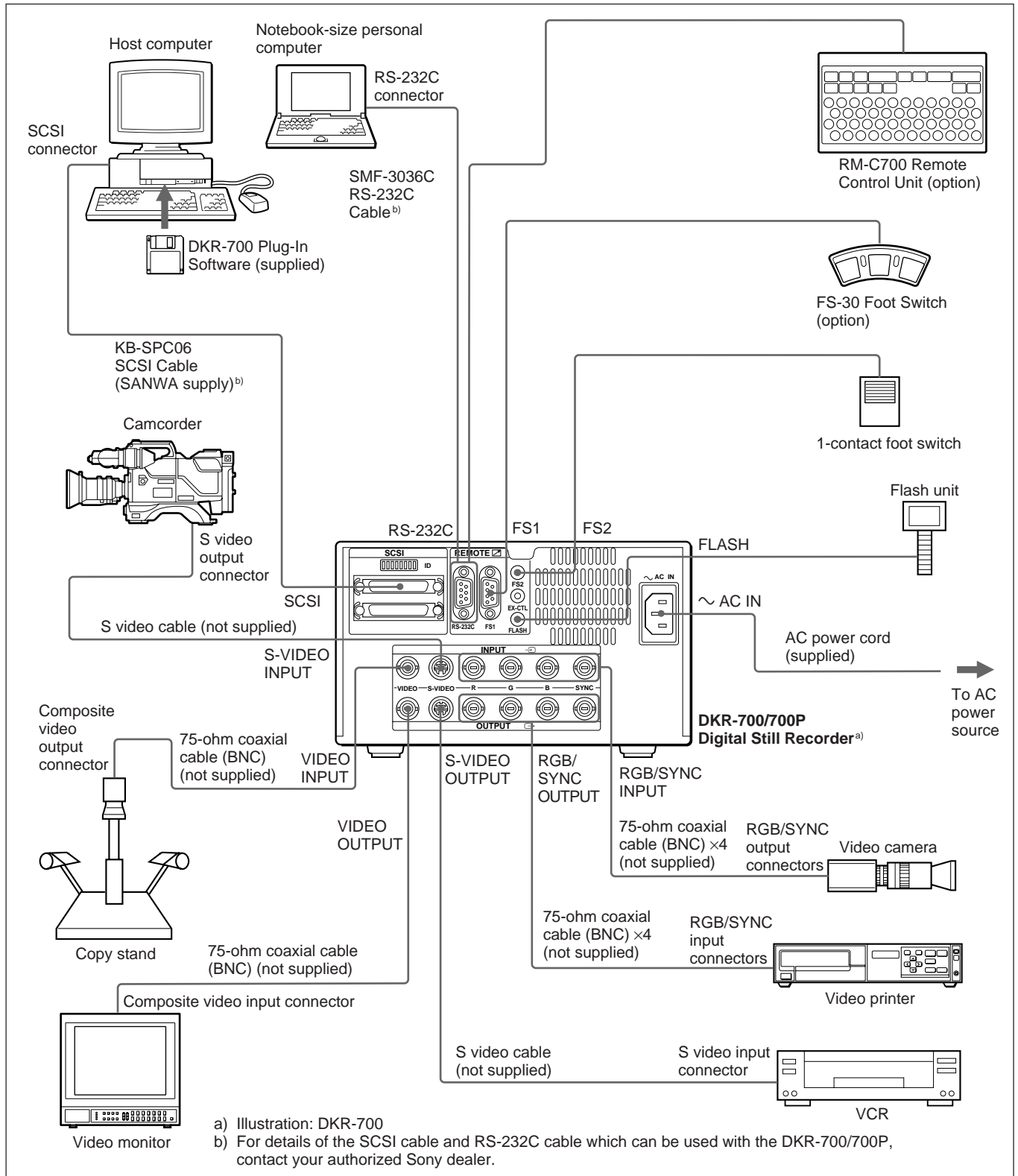

Setting the DIP Switch

Set the DIP switch as required. (Set the SCSI-ID when using the SCSI interface.)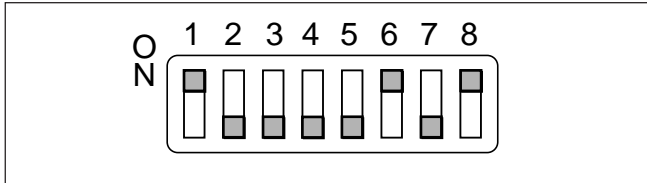

DIP switch

Setting and function of each switch

Switch number	Function	Setting
1	SCSI connector termination	To enable internal SCSI termination, set to ON.
2	Select the timing for capturing an input image to memory using a foot switch connected to FS1 connector or FS2 connector.	See table below.
3	Select the timing for capturing an input image to memory using a foot switch connected to FS1 connector or FS2 connector.	See table below.
4	reserved	Set to OFF.
5	reserved	Set to OFF.
6	SCSI-ID	See SCSI-ID Settings table.
7	SCSI-ID	See SCSI-ID Settings table.
8	SCSI-ID	See SCSI-ID Settings table.

Timing settings for image capture

Switch 2	Switch 3	Timing for image capture to memory
OFF	OFF	Third field after trigger signal is input. (Timing 1) ^{b)}
OFF	ON	Second field after trigger signal is input. (Timing 2) ^{b)}
ON ^{a)}	OFF ^{a)}	First field after trigger signal is input. (Timing 3) ^{b)}

a) When switch 2 is set to “ON” and switch 3 to “OFF”, then even when the Flash item in the System Setup menu is set to On, the signal is not output from the FLASH connector.

b) For more information about the timings 1 to 3, see “Timing for Capturing an Image Using a Foot Switch” (page 53(E)).

SCSI-ID Settings


SCSI-ID	0	1	2	3	4	5	6	7
Switch 6	OFF	OFF	OFF	OFF	ON	ON	ON	ON
Switch 7	OFF	OFF	ON	ON	OFF	OFF	ON	ON
Switch 8	OFF	ON	OFF	ON	OFF	ON	OFF	ON

MiniDiscs (MD Data Discs)

The DKR-700/700P uses data MiniDiscs (MMD-140 MD Data Disc or equivalent).

Using MiniDiscs

- Do not open the protective shutter. Touching the disc surface directly may damage or dirty the surface, causing errors or making data unreadable.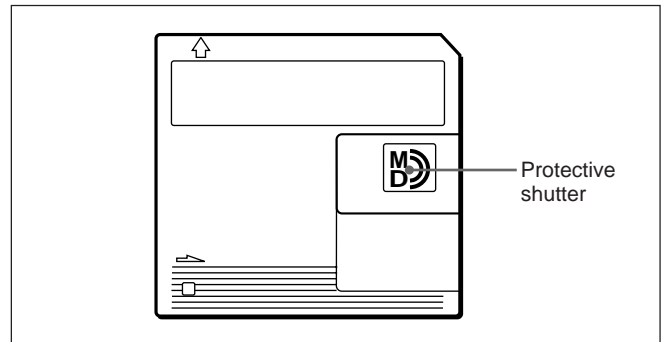


- Do not expose the discs to direct sunlight, elevated temperatures, or dusty environments.
- There is no need to clean MiniDiscs.

Preparation

Protecting against accidental erasure


When more data is written to a previously recorded (full) disc, the older data may be lost. To guard against accidentally erasing important data, slide the write protect switch as shown in the illustration.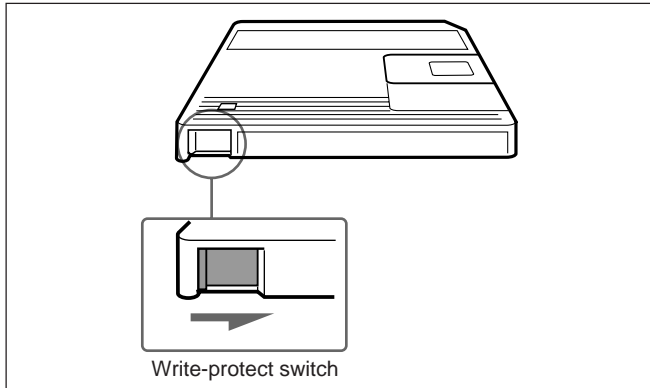

When a write-protected disc is inserted into the DKR-700/700P, recording operations will not be able to be performed. In order to re-record a disc, return the write protect switch to its original position.


Labeling discs

Attach the supplied label to the disc, and make a note of the disc's contents. If the label is full, remove the old label and affix a new one. If new labels are affixed on top of old labels, or if the label is not aligned in its proper position, **malfunctions may occur, and the disc drive may be damaged.**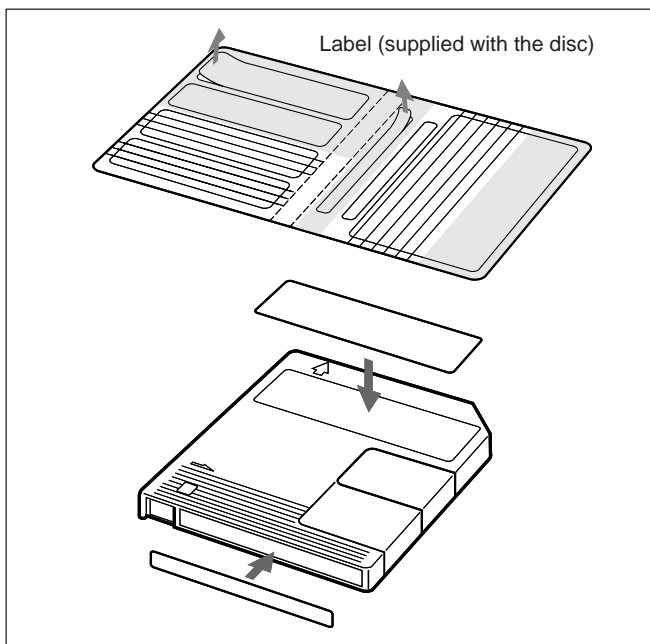

18(E)

Activating the DKR-700/700P and Inserting/Removing Discs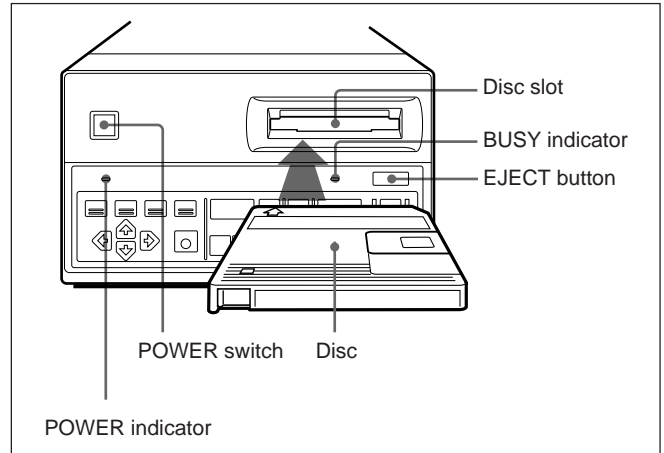

Activating the DKR-700/700P

Turn the POWER switch on.

The POWER indicator lights and the DKR-700/700P starts.

After a few seconds, the SOURCE button lights, and the keys on the front panel go into effect. The DKR-700/700P beeps if the "Beep" in the System Setup menu is set to On.

For information on enabling the beep sound, see page 46(E).

The POWER switch may be turned on or off while a disc is loaded.

Inserting a disc

Holding a disc with the arrow side up, insert it in the slot in the direction of the arrow.

Note

The orientation of a MiniDisc is different from that of a floppy disk or MO disc.

When activating the DKR-700/700P, the BUSY indicator lights and the DKR-700/700P starts to check the data on the disc.

When data check completes, the BUSY indicator goes out, and the DKR-700/700P beeps if the “Beep” in the System Setup menu is set to On.

If the disc is not initialized or initialized to a format incompatible with the DKR-700/700P

The ALL ERASE/INIT button blinks (the message “Initialize” appears on the screen), and the DKR-700/700P stands-by for initialization. The button will stop blinking if it is left for about five seconds.

See “Initializing Discs” on this page for details.

Removing the disc

Press the EJECT button to eject the disc.

If the BUSY indicator is lit or blinking when you press the EJECT button, the disc is ejected after internal processing is completed.

The Disc may be stored in the DKR-700/700P except when shipping the DKR-700/700P.

Initializing Discs


To record images on a disc with the DKR-700/700P, the disc must first be initialized to a format compatible with the DKR-700/700P.

Note

Initializing a disc erases all data in the disc. Check the data recorded on the disc before beginning the operation.

- 1 Insert a disc to be initialized into the DKR-700/700P and activate the DKR-700/700P (see page 18(E)).

The ALL ERASE/INIT button blinks (the message “Initialize” appears on the screen).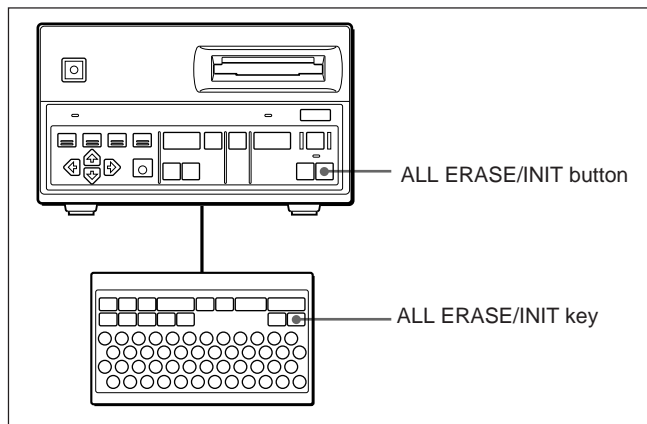

- 2 While the ALL ERASE/INIT button is blinking, press it (or the ALL ERASE/INIT key). If the ALL ERASE/INIT button/key is left for about five seconds, the button will stop blinking. The button will start blinking by pressing it (or the ALL ERASE/INIT key).

The ALL ERASE/INIT button lights, and disc initialization starts.

After about 3 minutes, initialization completes (the ALL ERASE/INIT button goes out).

Caution

During initialization the BUSY indicator is lit or blinking. At this point, be careful not to disconnect the AC power cord from the outlet. If the power is disconnected **the disc could be damaged.**

Setting the Clock


Before beginning to use the DKR-700/700P, set the internal clock to the proper date and time.

The clock is set using the System Setup menu. For more information, see page 46(E).

Preparation

Checking the Setup/Operating Condition of the DKR-700/700P

The DKR-700/700P's setup information and operating conditions can be displayed on the screen.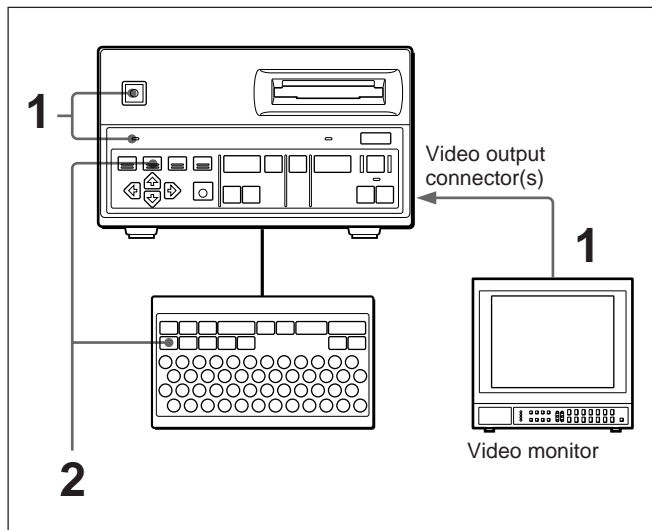


- 1** Connect a video monitor to the video output connector(s) on the rear panel of the DKR-700/700P (see page 16(E)) and activate the DKR-700/700P (see page 18(E)).
- 2** Press the DISP button/key.

Operation and setup information appears.

Note

If the “Live” item in the Capture/Rec Setup menu is set to Thru, operation and setup information will not be displayed when the DISP button/key is pressed.

For more information on the Capture/Rec Setup menu, see page 41(E).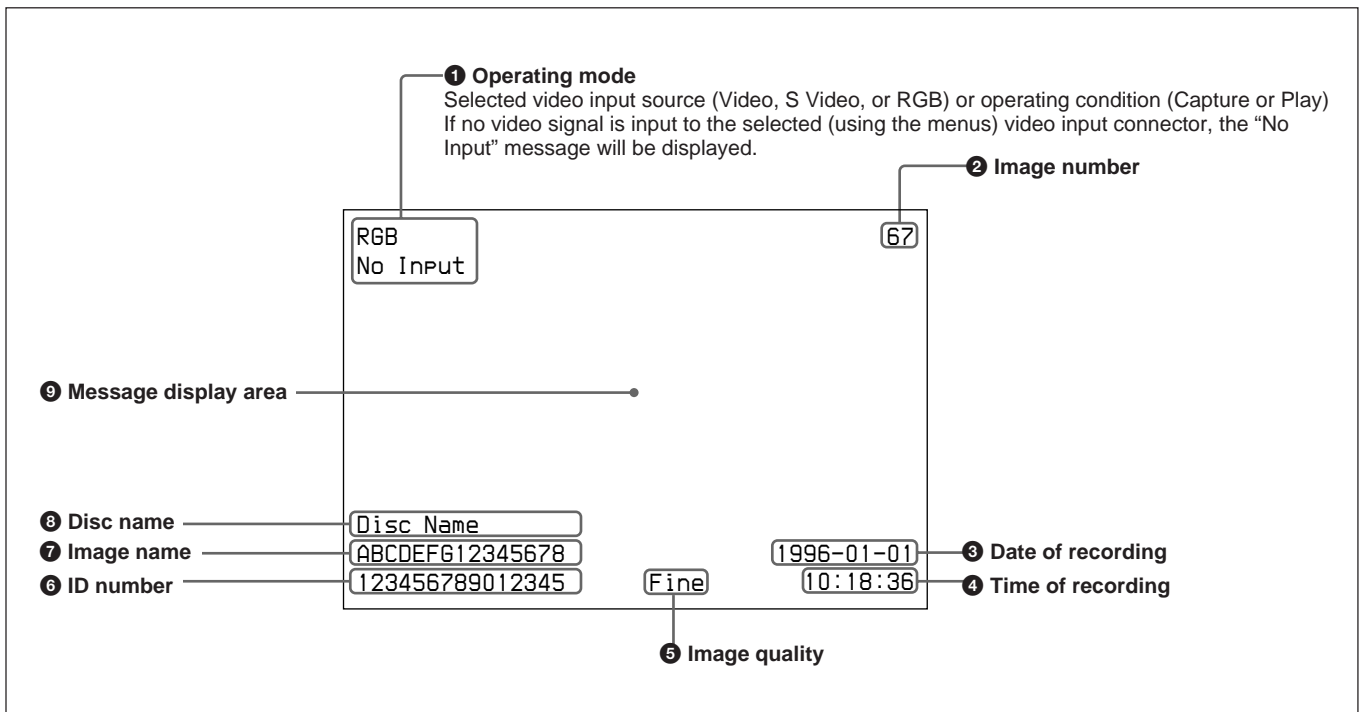

Items **2** to **8** change with the operating condition, as follows:

When playing: Information about the image currently displayed on screen.

Otherwise: Information about settings to be used for the next image recorded. Item **3** is not displayed. At the item **4** position the number of images the disc can still accommodate is displayed.


Changing the display format

Display of each item can be set on or off using the Disp Setup menu.

For more information on setting the display, see page 44(E).

Recording Images

Preparing to Record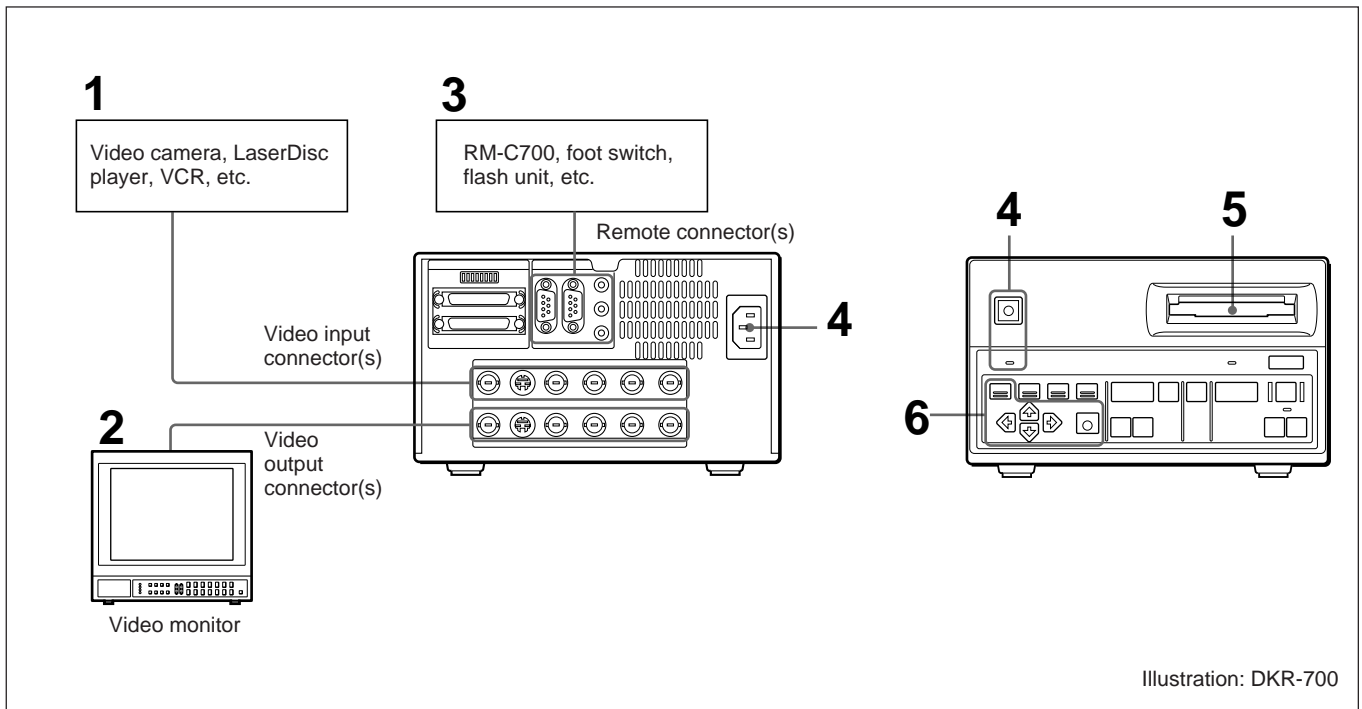

- 1 Connect a video camera, LaserDisc player, VCR, etc. to the proper video input connector(s) on the rear panel (*see page 16(E)*).
- 2 Connect a video monitor to the proper video output connector(s) on the rear panel (*see page 16(E)*).
- 3 Connect necessary auxiliary devices (RM-C700, foot switch, flash unit, etc.) to the proper remote connector(s) on the rear panel (*see page 16(E)*).
- 4 Power the DKR-700/700P (*see page 18(E)*).

Warning

When using the DKR-700P, make sure that the voltage setting meets the voltage in the mains line prior to the operation.

- 5 Insert a disc into the disc slot, after verifying that the disc has been properly initialized and that the write-protect switch is set to allow recording (*see page 18(E)*).

- 6 Using the menus, make the following settings:

- 1) On the main menu, set “Input” appropriately for the input video signal to be used (*see page 39(E)*).
- 2) Set each item in the Source Setup menu appropriately for the input video signal to be used (*see page 40(E)*).
- 3) Set the necessary items on the System Setup menu (*see page 45(E)*) and Capture/Rec Setup menu (*see page 41(E)*) appropriately for the external devices to be used.


Adding ID numbers or image names to images

When an image is recorded, its image numbers and recording date are automatically recorded as well. When using the RM-C700, ID numbers and image names can also be recorded.

For information about ID number and image name recording procedures, see “Image Data Management (Using the RM-C700)” on page 32(E).

Recording One Image at a Time — Step Recording

This section describes the operating procedures to be followed using either the DKR-700/700P front panel or the RM-C700.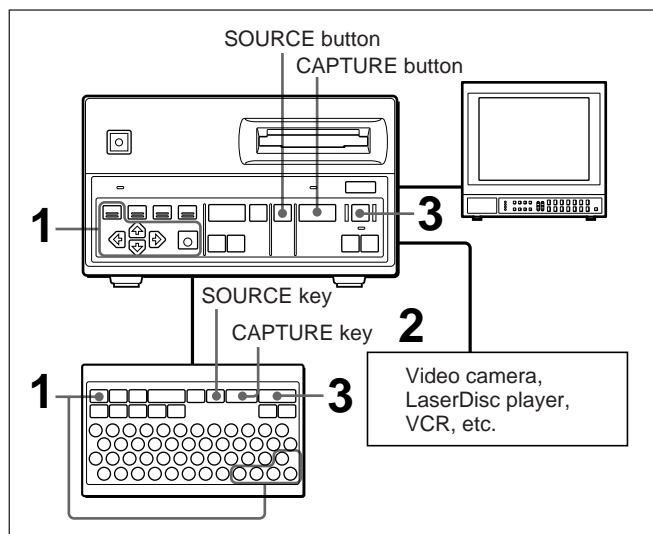

- 1 In the Capture/Rec Setup menu, set the following items:
 - Quality:** Select the desired image quality.
 - Rec Mute:** To reduce recording times, set this to On.
 - Flush:** To record a number of consecutive images at short intervals, set this to a number other than 0.
 - Auto Rec:** Off
 - Overwrite:** To allow recording over pre-recorded images when the disc is full, set to On.

Note

When the “Flush” item is set to a value other than 0, after image recording is completed, when the specified time has elapsed the image information recording begins. During this interval, be careful not to disconnect the AC power cord from the outlet.

For menu settings, see page 41(E).

- 2 Send a video signal to the DKR-700/700P from the video camera, LaserDisc player, VCR, etc.

- 3 When the image to be recorded is displayed on the screen, press the REC button/key.

The image is frozen on the screen, and recording to the disc begins (the REC button lights).

Note

When operating the unit with an input signal from a VCR, do not carry out variable-speed operations (fast forward, rewind, jog/shuttle playback, etc.) on the VCR, as this may cause the frozen picture to break up.

Caution

While recording the BUSY indicator is lit or blinking. At this point, be careful not to disconnect the AC power cord from the outlet. If the power is disconnected **the disc could be damaged.**

Confirming the image before recording

- 1 Press the CAPTURE button/key instead of the REC button/key in step 3.

The image is frozen on the screen, and captured in the DKR-700/700P’s memory.

If the captured image is not the image desired

Press the SOURCE button/key to terminate the freeze mode and perform step 1.


- 2 Press the REC button/key.

The image stored in memory is recorded to the disc.

Recording Images

Recording Images Automatically — Interval Recording

This section describes the operating procedures to be followed using either the DKR-700/700P front panel or the RM-C700.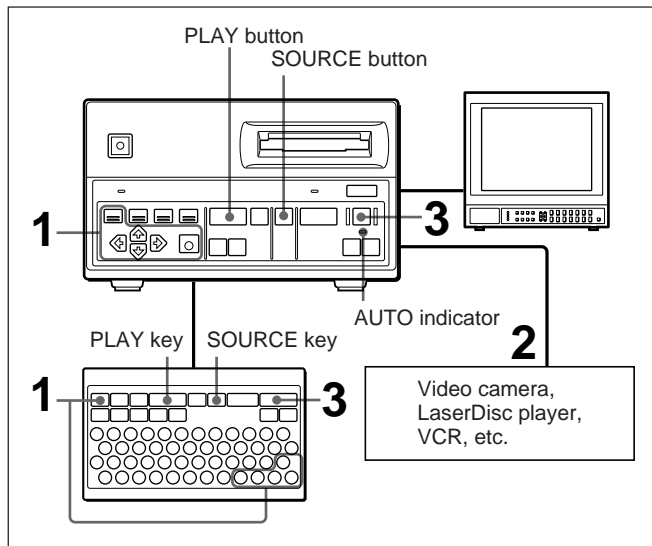

- 1 In the Capture/Rec Setup menu, set the following items:
 - Quality:** Select the desired image quality.
 - Rec Mute:** To reduce recording times, set this to On.
 - Flush:** To record a number of consecutive images at short intervals, set this to a number other than 0. (See “Note” in step 1 on page 23(E).)
 - Auto Rec:** Off
 - Overwrite:** To allow recording over pre-recorded images when the disc is full, set to On.

For menu settings, see page 41(E).

- 2 Send a video signal to the DKR-700/700P from the video camera, LaserDisc player, VCR, etc.
- 3 Press the REC button/key.

The image is frozen on the screen, and recording to the disc begins (the AUTO indicator lights).

Note

When operating the unit with an input signal from a VCR, do not carry out variable-speed operations (fast forward, rewind, jog/shuttle playback, etc.) on the VCR, as this may cause the frozen picture to break up.

To stop recording

Press the SOURCE button/key or PLAY button/key.

Recording with a 1-Contact Foot Switch

One of the following three modes can be selected for a 1-contact foot switch using the Capture/Rec Setup menu:

- Capture/Rec
- Capture
- Alternate

For menu settings, see page 41(E).

To record with the Capture/Rec mode

- 1 Perform steps 1 and 2 of “Recording One Image at a Time — Step Recording” (page 23(E)).
- 2 Press the foot switch when the image to be recorded is displayed on the screen.

The image is frozen on the screen, and recording to disc begins.

To record with the Capture mode

- 1 Perform steps 1 and 2 of “Recording One Image at a Time — Step Recording” (page 23(E)).
- 2 Press the foot switch when the image to be recorded is displayed on the screen.

The image is frozen on the screen, and captured in the DKR-700/700P’s memory.

If the captured image is not the image desired
Repeat pressing the foot switch until the desired image is captured.

- 3 Press the REC button on the DKR-700/700P or the REC key on the RM-C700 .

The image stored in memory is recorded to the disc.

To record with the Alternate mode

1 Perform steps **1** and **2** of “Recording One Image at a Time — Step Recording” (page 23(E)).

2 Press the foot switch when the image to be recorded is displayed on the screen.


The image is frozen on the screen, and captured in the DKR-700/700P’s memory.

3 Press the foot switch again.

The image stored in memory is recorded to the disc.

Recording with the Optional FS-30 Foot Switch

With the optional FS-30 Foot Switch you can select any of three operating modes A, B, and C.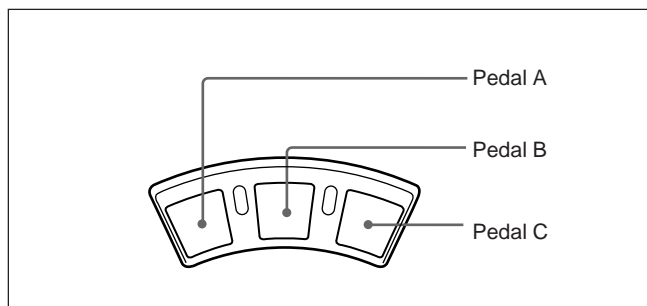

		Pedal		
		A	B	C
Mode	A	Play	Source	Capture/Rec
	B	Play	Rec	Capture
	C	Rec	Source	Capture

Play: Play back the most recently recorded image.

Source: Same function as the SOURCE button on this unit or on the RM-C700.

Capture: Capture the desired image to memory.

Rec: If pressed after executing a Capture, record the image captured to memory on the disc.


If pressed other than after executing a Capture, capture the desired image to memory, then record it on the disc.

Capture/Rec: Capture the desired image to memory, then record it on the disc.

For more information about recording operations, see the procedures under “Recording One Image at a Time — Step Recording” (page 23(E)).

Playing or Searching for Images

Preparing to Playback or Search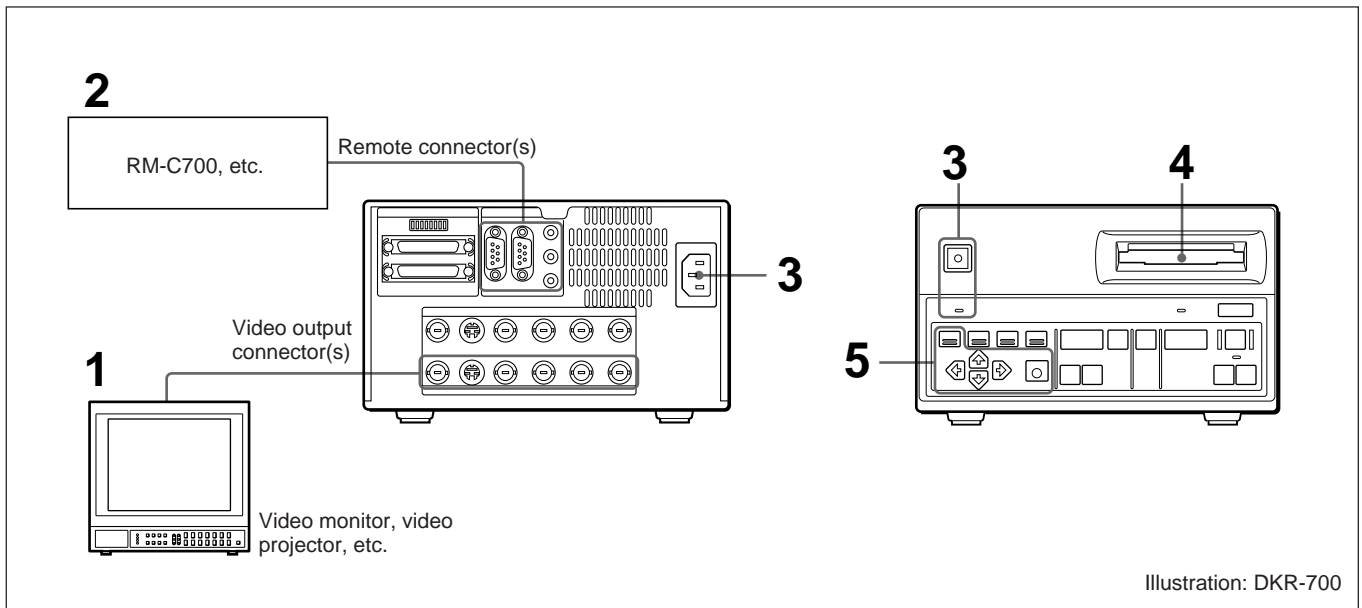

- 1 Connect a video monitor, video projector, etc., to the proper video output connector(s) on the rear panel of the DKR-700/700P (see page 16(E)).
- 2 Connect necessary auxiliary devices (RM-C700, etc.) to the proper remote connector(s) on the rear panel (see page 16(E)).
- 3 Power the DKR-700/700P (see page 18(E)).

Warning

When using the DKR-700P, make sure that the voltage setting meets the voltage in the mains line prior to the operation.


- 4 Insert a disc containing the images into the disc slot (see page 18(E)).
- 5 1) In the main menu, set the “Video” appropriately for the output video signal to be used (see page 39(E)).
2) Set the necessary items in the System Setup menu appropriately for the external devices to be used (see page 45(E)).

Note

When operating the unit with an input signal from a VCR, do not carry out variable-speed operations (fast forward, rewind, jog/shuttle playback, etc.) on the VCR, as this may cause the playback picture to break up.

Playing One Image at a Time — Step Playback

This section describes the operating procedures to be followed using either the DKR-700/700P front panel or the RM-C700.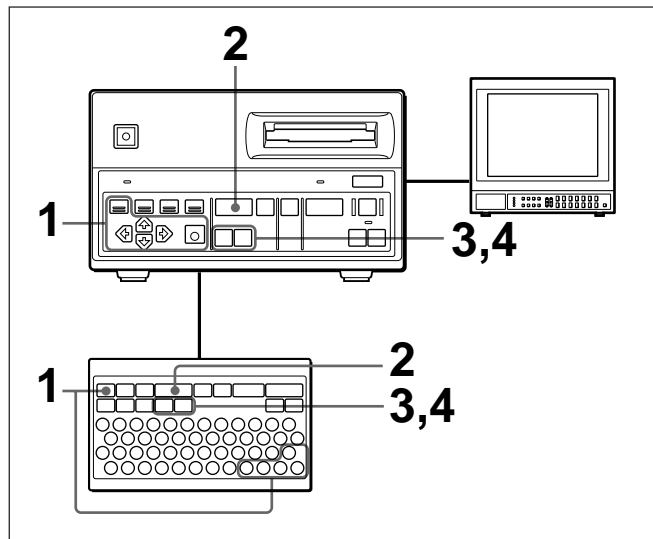

- 1 Make the following settings in the Play Setup menu.

“Mode” setting

When high-quality is demanded (main image display): Select Main.

When quick access is demanded (quick access image display): Select Quick.

If Auto is selected, the both quick access image and main image are played back.

On the DKR-700P, playing back a quick access image results in a black frame around the image.

Note

When playing back a main image (i.e. either Main or Auto was selected in step 1), and when the image to be displayed was stored uncompressed (by setting the “Quality” to Non Comp in the Capture & Rec Setup menu), it may take a few moments for the image to be displayed. (During this time, no image will appear on the screen.)

“Play Mute” setting

To reduce the time until the image appears: Set this to “On.” When the image is changed the screen will go blank (i.e. be “muted”).

If you do not wish to blank (i.e. “mute”) the image changes: Set this to “Off.” The time taken for the image to appear will be longer than when you use the “On” setting.

For menu settings, see page 43(E).

- 2 Press the PLAY button/key.

The DKR-700/700P starts playback (the PLAY button lights).

If you selected “Quick” in step 1, the quick access image appears, and if you press the PLAY key once more during playback the main image appears.

- 3 Press the \square or \square button/key.

When the \square button/key is pressed: An image with larger-number is displayed.

When the \square button/key is pressed: An image with smaller-number is displayed.

Note


If there are images already recorded on the disc with a different type of device, the image numbers may in some cases not be consecutive.

- 4 Repeat step 3 to display other images.

Playing or Searching for Images

Playing Images Automatically — Interval Playback

This section describes the operating procedures to be followed using either the DKR-700/700P front panel or the RM-C700.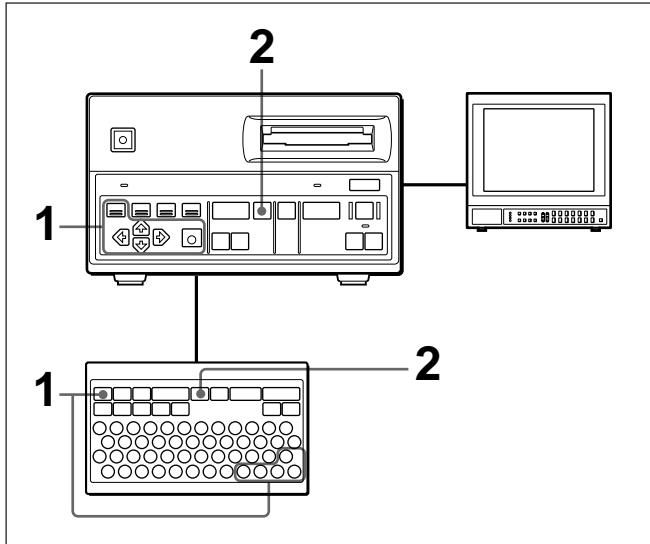

- 1 Set the following items in the Play Setup menu:
 - Mode:** See step 1 in the previous section “Playing One Image at a Time — Step Playback”.
 - Play Mute:** See step 1 in the previous section, and make the appropriate setting. (“Mute” means “blank.”)
 - Auto Play**
 - Interval:** Set to the desired interval.
 - Direction:** Select the desired direction.


For menu settings, see page 43(E).

- 2 Press the AUTO PLAY button/key.

Images are displayed in the order and at the interval specified in the menu settings.

Playing an Image by Its Image Number — Direct Access Playback (Using the RM-C700)

Direct access playback is performed using the RM-C700.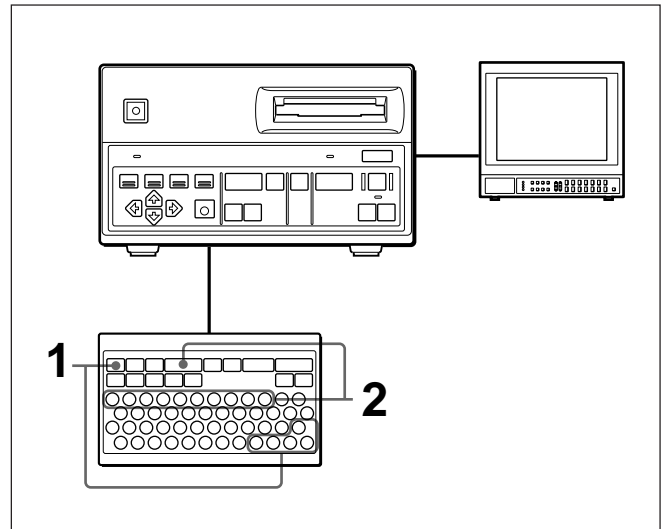

- 1 Set the “Mode” in the Play Setup menu to the desired mode.


See step 1 in the section “Playing One Image at a Time — Step Playback” on page 27(E).

- 2 Enter the number of the desired image and press the PLAY button/key.

The selected image is displayed on the screen.


Searching for an Image Using the Image Index

This section describes the operating procedures to be followed using either the DKR-700/700P front panel or the RM-C700.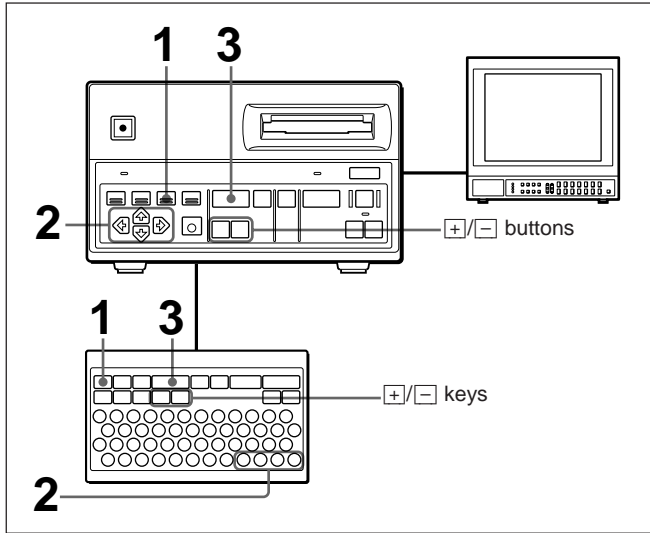


- 1 Press the IMAGE button/key.

The first page of the image index is displayed on the screen.

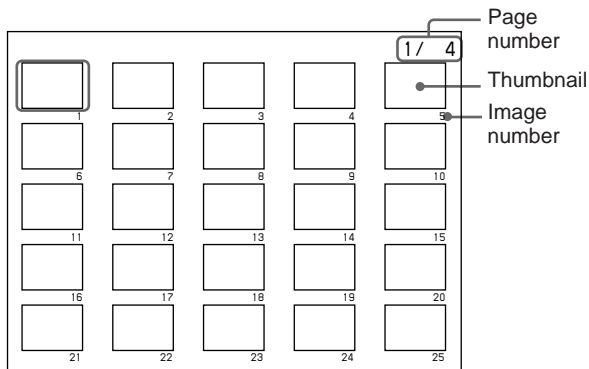- 2 Using the cursor buttons/keys, select the desired image.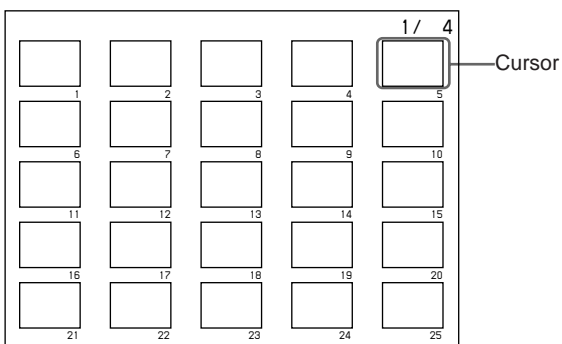

If the desired image is not found in the first page


Press the \square or \square button/key to change pages.

- 3 Press the PLAY button/key.

The selected image is displayed on the screen.


Searching for an Image Using Information Search List

Using the front panel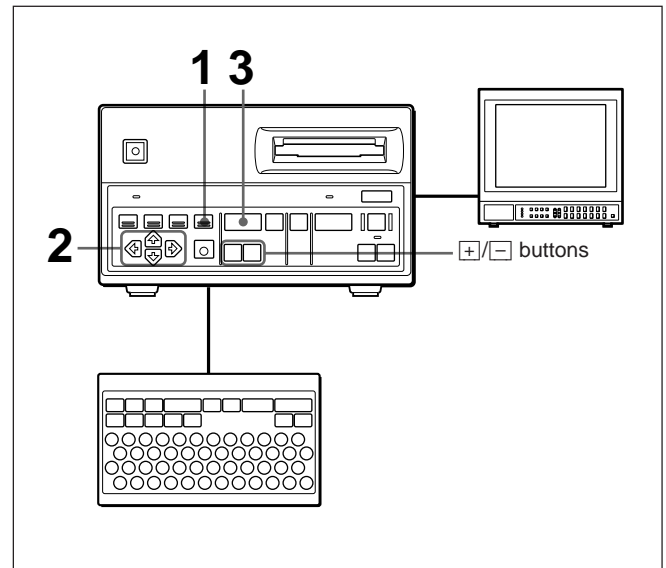

- 1 Press the LIST button.

The first page of the information search list is displayed on the screen.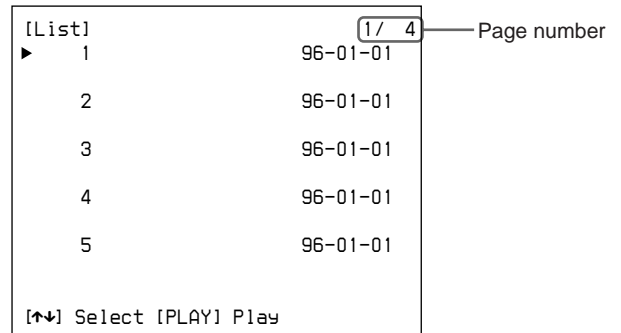

(Continued)

Playing or Searching for Images

- 2** Using the or button, select the desired image.

[List]		1 / 4
1	96-01-01	
2	96-01-01	
3	96-01-01	
▶ 4	96-01-01	
5	96-01-01	

[↑↓] Select [PLAY] Play


If the desired image is not found

Press the or button to change pages.

- 3** Press the PLAY button.

The selected image is displayed on the screen.

Using the RM-C700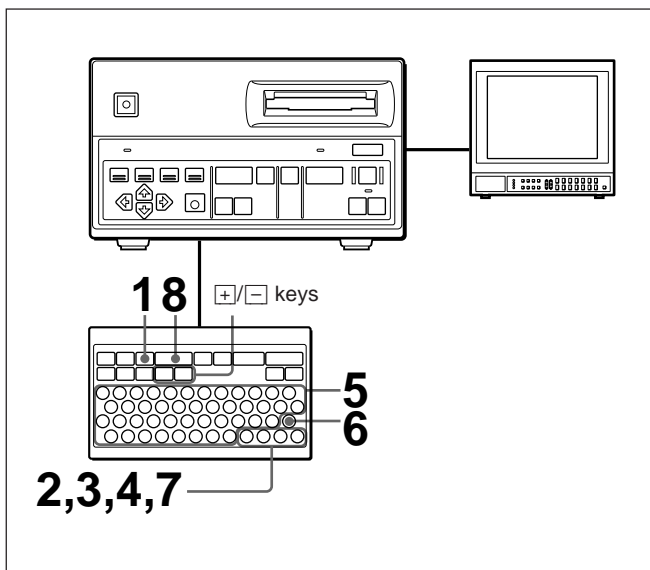

- 1** Press the LIST key.

The List Search display appears on the screen.

[List Search]

Search: ID No

Keyword:

[↑↓] Select [↔] Set [EXEC]Execute

Cursor

- 2** Using the or key, move the cursor (▶) to the "Search" line.

- 3** Use the or key to select the type of search to be performed.

The following selections are possible:

Image No (image number), Title (image name), Date, and ID No (ID number).

- 4** Using the or key, move the cursor (▶) to the "Keyword" line, and press the or key to display the cursor (▬).

[List Search]

Search: ID No

▶Keyword: ▬

[↑↓] Select [↔] Set [EXEC]Execute

Cursor

- 5** Depending on the selection made in step 3, enter the image number, image name, date, or ID number of the image to be searched for.

[List Search]

Search: ID No


▶Keyword: 123456_

[↑↓] Select [↔] Set [EXEC]Execute

There is no need to enter all numbers or characters. Search is possible with several numbers or characters from the head of the ID number or image name.

6 Press the EXE key.


An information search list containing the information about the desired image is displayed on the screen.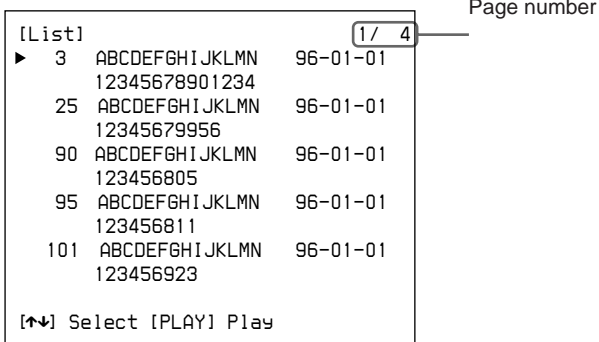

If the entry for the desired image is not on the list

Press the **[+]** or **[-]** key to change pages.

7 Using the **[↑]** or **[↓]** key, move the cursor to the number of the image to be displayed.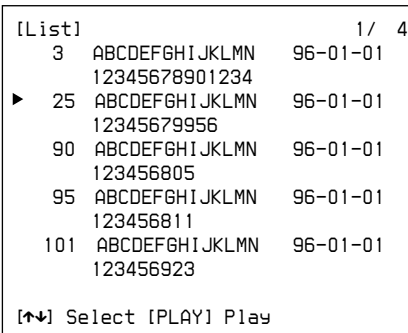

8 Press the PLAY key.


The selected image is displayed on the screen.

Image Data Management (Using the RM-C700)

ID numbers and image names can be associated with image data, and discs may be assigned names, using the optional RM-C700 Remote Control Unit.

Keys Used to Input ID Numbers, Image Names and Disc Names

The following keys are used for the input of ID numbers, image names and disc names: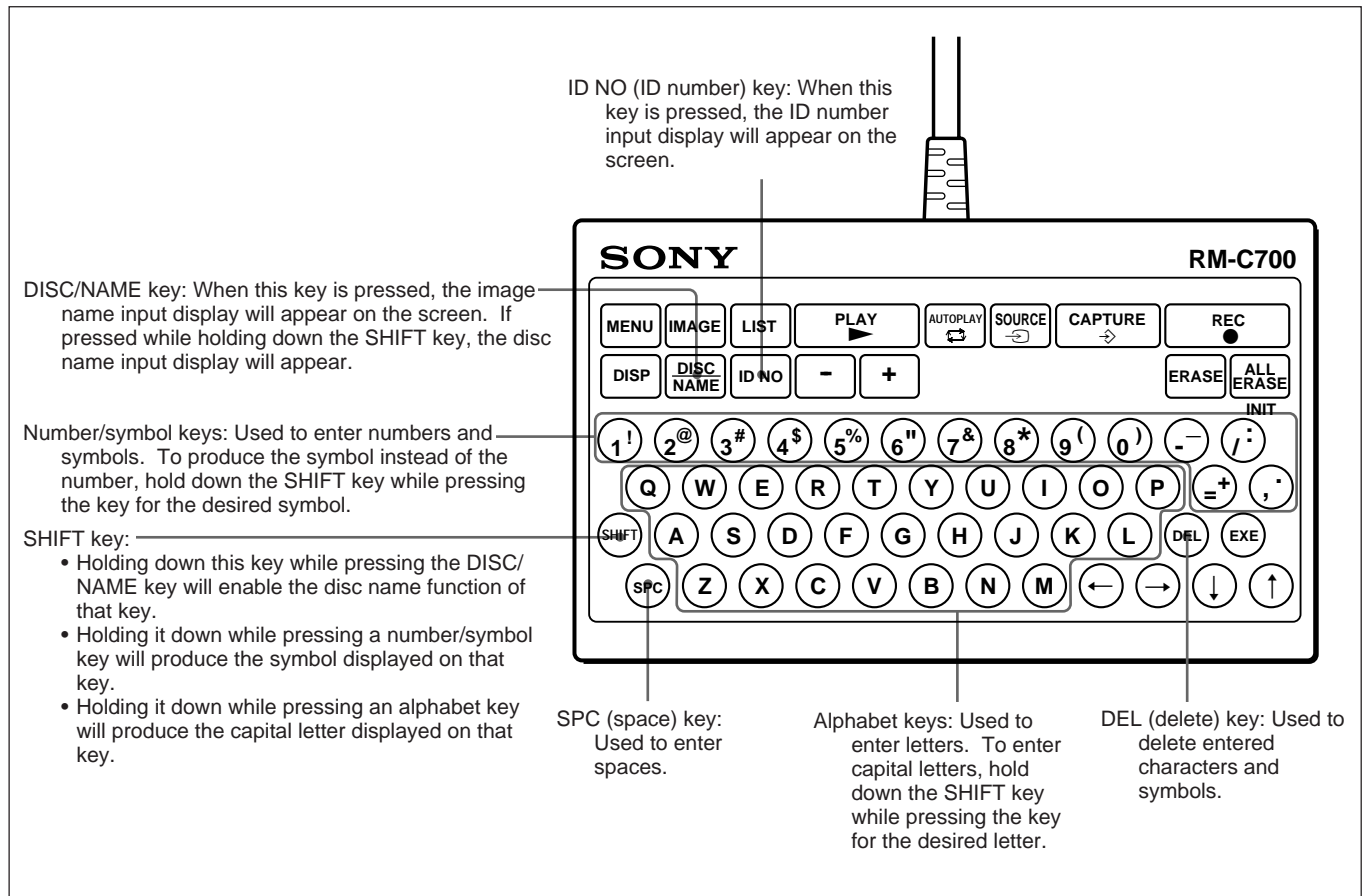

Setting/Changing the Disc Name

- 1) Connect a video monitor to the appropriate connector(s) on the rear panel of the DKR-700/700P (see page 16(E)).
2) Insert the disc whose name is to be added or modified into the DKR-700/700P, after making sure that it is not write-protected (see page 18(E)).
- 2 Press the DISC/NAME key while holding down the SHIFT key.

The disc name input display appears.

```
Disc Name:
[↔] Select [EXE] End
```

- 3 Press the or key to display the cursor and move it to the position where characters are to be entered or changed.

```
Disc Name: ● Cursor
[↔] Select [EXEC] End
```

- 4 Enter the desired character (up to 15 characters).

```
Disc Name:Photo_
[↔] Select [EXEC] End
```

The entered character is displayed, the cursor moves one position to the right.

- 5 Check the entered name, and press the EXE key if it is correct.

Note

Once a disc name is set, it can not be erased.

Setting/Changing an ID Number

Setting an ID number

- 1 Press the ID NO key.

The ID number input display appears.

```
▶ID No:
Mode:Off
[↔] Select [↔] Set
```

- 2 Using the or key, move the cursor (▶) to the "ID No" line.

- 3 Press the or key to display the cursor (●).

```
▶ID No: ● Cursor
Mode:Off
[↔] Select [↔] Set
```

(continued)

Image Data Management (Using the RM-C700)

4 Enter the desired number (up to 15 digits).

Each time a digit is entered, the cursor will move one position to the right.

```
▶ID No:13479_
Mode:Off

[←→] Select [EXEC] End
```

5 Using the \uparrow or \downarrow key, move the cursor to the "Mode".

```
ID No:13479_
▶Mode:Off

[↑↓] Select [←→] Set
```

6 Using the \leftarrow or \rightarrow key, change the setting to Hold or Inc.

Hold: Use this ID number for all images recorded until a new ID number is set.

Inc: Each time a new image is recorded, increment the ID number and attach it to that image.

To disable recording of ID numbers

Set the "Mode" to Off.

7 Check the entered ID number and the Mode setting, and press the EXE key if they are correct.

Note

The ID set is lost when the unit is powered off.

Modifying a recorded ID number

1 Play back the image whose ID number is to be changed, using the procedures in "Playing or Searching for Images" (page 26(E)).

Notes

- Be sure the disc is not write-protected.
- Play back the image with some mode other than auto play. ID numbers may not be changed while in auto play mode.

2 Press the ID NO key.

The ID number rewrite display appears.

```
[Rewrite]
ID No:12345678

[←→] Select [EXEC] End
```

3 Press the \leftarrow or \rightarrow key to display the cursor and move it to the position where digit is to be changed.

```
[Rewrite]
ID No:12345678

[←→] Select [EXEC] End
```

4 Enter the desired digit.

```
[Rewrite]
ID No:12345978

[←→] Select [EXEC] End
```

5 Press the EXE key.

Note


Once an ID number has been recorded, it can be changed, but cannot be erased.

Setting/Changing an Image Name


Setting an image name

- 1 Press the DISC/NAME key.

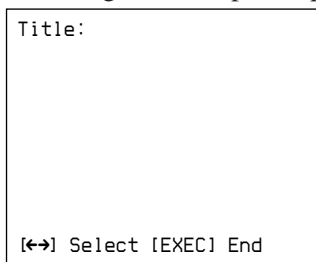The image name input display appears.


- 2 Press the  or  key to display the cursor (|).

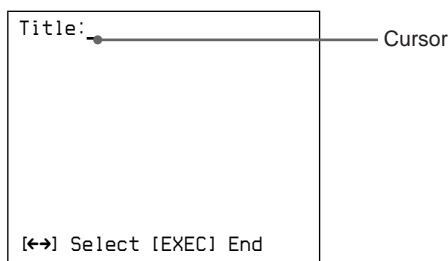- 3 Enter the desired character (up to 15 characters).

Each time a character is entered, the cursor will move one position to the right.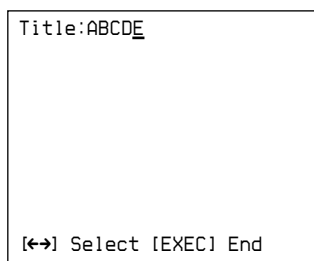

- 4 Check the entered image name, and press the EXE key if it is correct.

Note

The image name set is lost when the unit is powered off.

Modifying a recorded image name


- 1 Play back the image whose image name is to be changed, using the procedures in “Playing or Searching for Images” (page 26(E)).

Notes


- Be sure the disc is not write-protected.
- Play back the image with some mode other than auto play. Image names may not be changed while in auto play mode.

- 2 Press the DISC/NAME key.

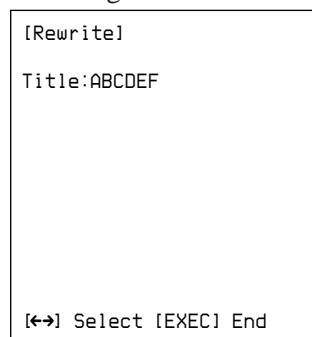The image name rewrite display appears.

- 3 Press the  or  key to display the cursor and move it to the position where character is to be changed.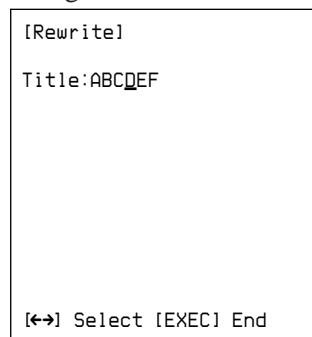

- 4 Enter the desired character.

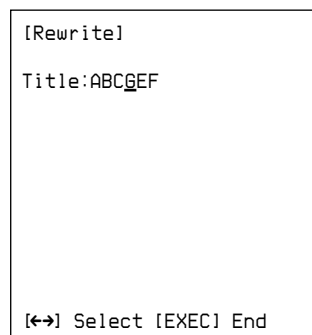
Image Data Management (Using the RM-C700)

5 Press the EXE key.

Note


Once an image name has been recorded, it can be changed, but cannot be erased.

Deleting Images

This section describes the operating procedures to be followed using either the DKR-700/700P front panel or the RM-C700.

Erasing Images One at a Time

- 1 Display the image to be erased. (See “Playing or Searching for Images” on page 26(E) for procedures.)
- 2 Press the ERASE button/key.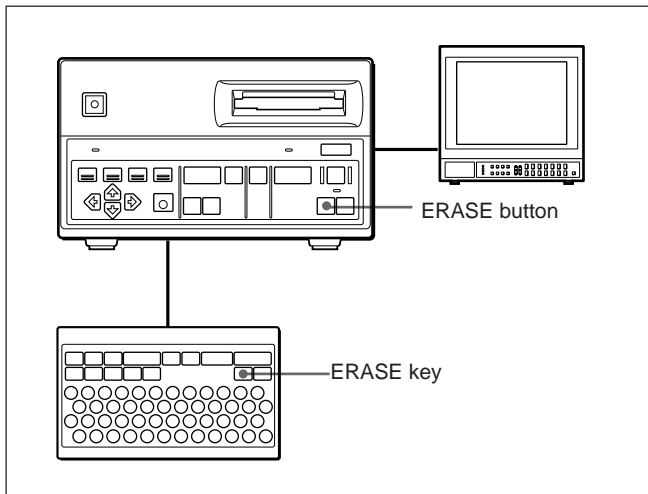

The ERASE button blinks, and the message “Erase” appears on the screen.

- 3 While the ERASE button is blinking, press it (or the ERASE key) again. If the ERASE button/key is left for about five seconds, the ERASE button will stop blinking. It will start blinking by pressing it (or the ERASE key) again.

The ERASE button lights, and the image disappears from the screen. The image number of a deleted image is omitted from the series.

Caution


During erase the BUSY indicator is lit or blinking. At this point, be careful not to disconnect the AC power cord from the outlet. If the power is disconnected, **the data on the disc may be damaged.**

Erasing All Images on the Disc

Note

Before beginning the operation, check the images recorded on the disc. Once images have been erased, there is no way to recover them.

- 1 Insert a disc into the DKR-700/700P (see page 18(E)).
- 2 Press the ALL ERASE/INIT button/key.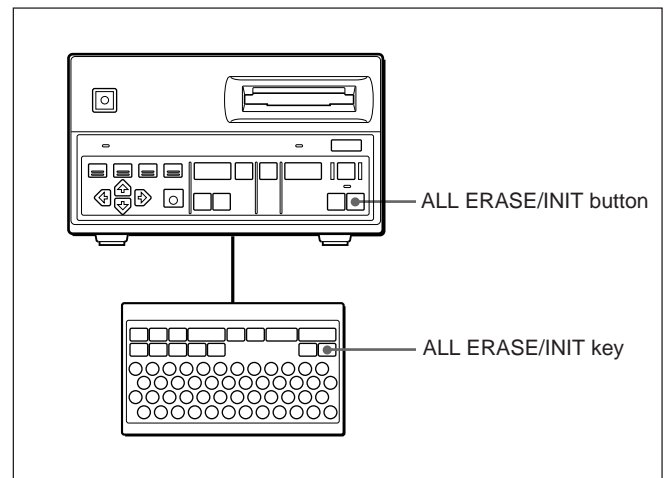

The ALL ERASE/INIT button blinks, and the message “ALL ERASE” appears on the screen.

- 3 While the ALL ERASE/INIT button is blinking, press it (or the ALL ERASE/INIT key) again. If the ALL ERASE/INIT button/key is left for about five seconds, the ERASE button will stop blinking. It will start blinking by pressing it (or the ALL ERASE/INIT key) again.

The ALL ERASE/INIT button lights, and all erase starts.

Caution


During erase the BUSY indicator is lit or blinking. At this point, be careful not to disconnect the AC power cord from the outlet. If the power is disconnected, **the data on the disc may be damaged.**

Menu Operations

The parameters dealing with unit operations and operating environment are set using menus.


Menu Structure

The main menu and six setup menus are provided as shown below (the figure is for the DKR-700).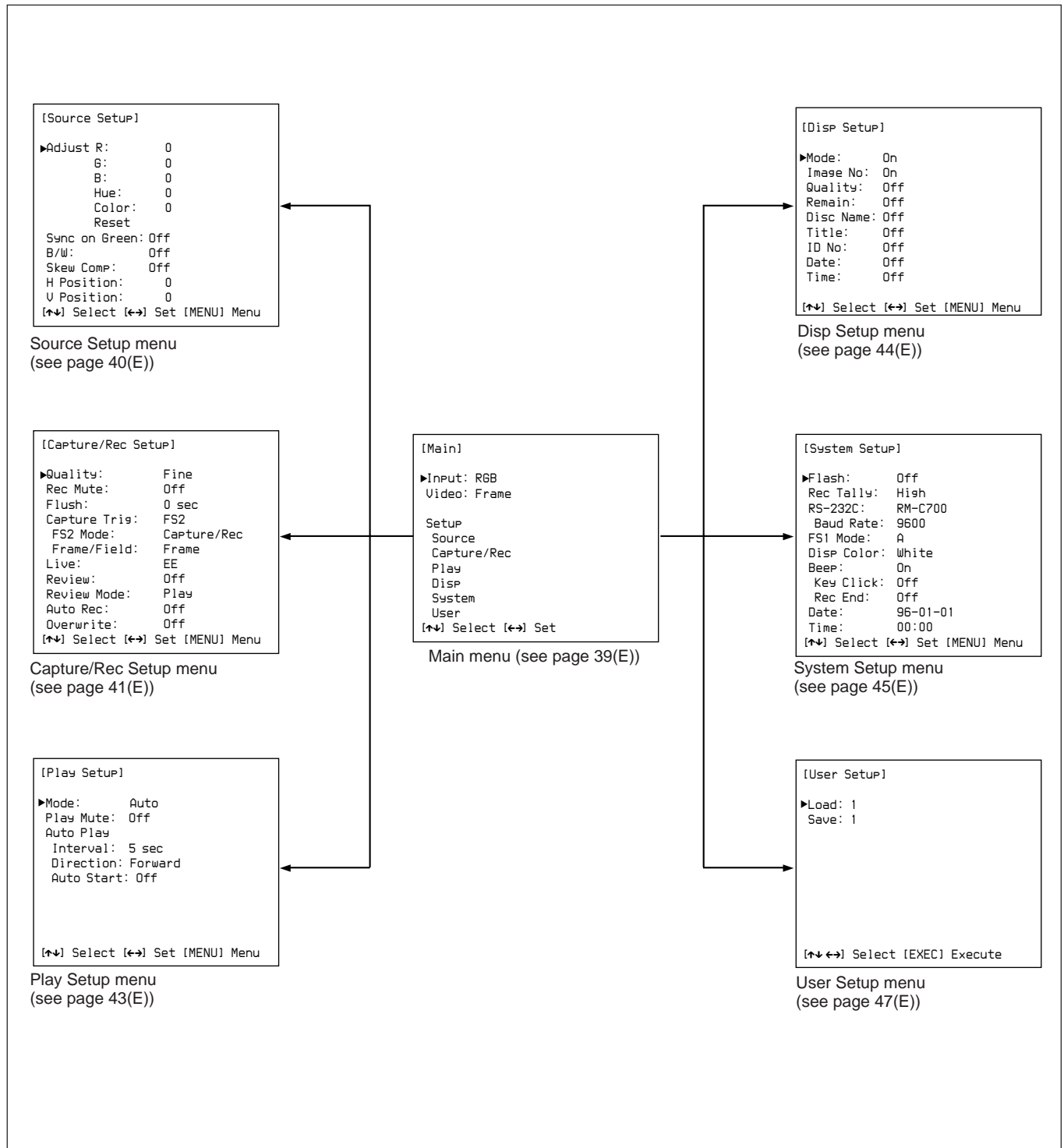

Buttons/Keys for Menu Operations

The buttons or keys shown below are used in menu operations.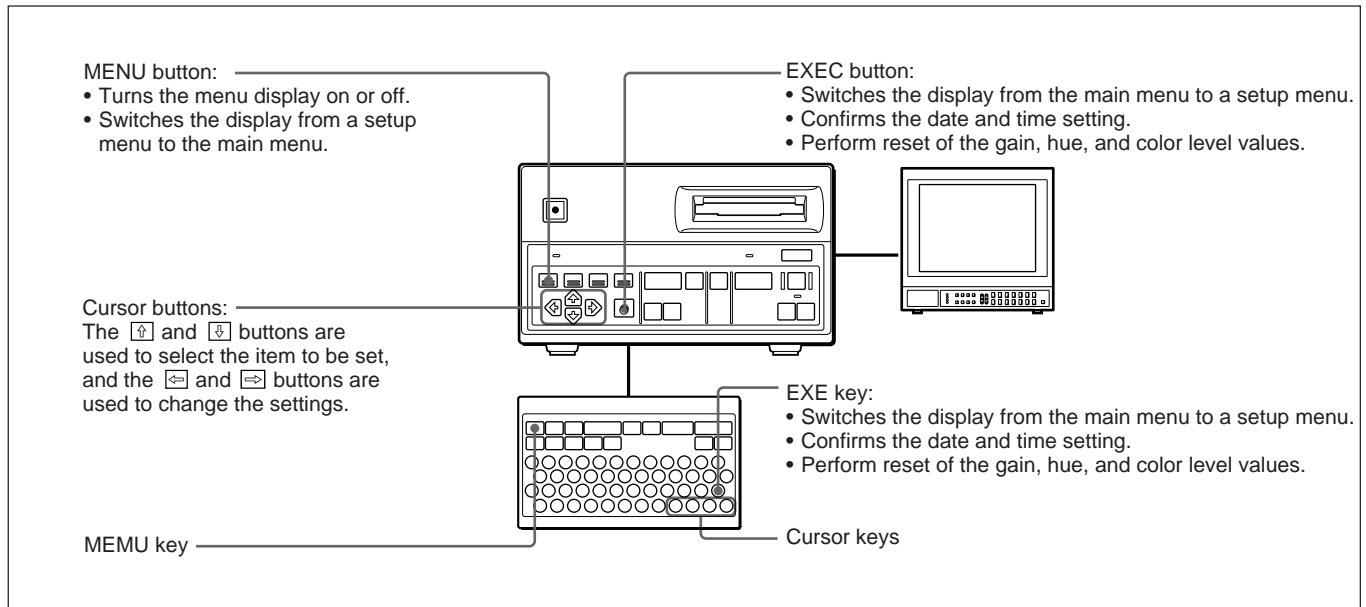

Structure and Usage of the Main Menu

Press the MENU button/key.

The main menu appears.

```
[Main]
▶Input: RGB
Video: Frame

Setup
Source
Capture/Rec
Play
Disp
System
User
[↑↓] Select [←→] Set
```

The details of each item are as follows:

Input: Selects the type of input video signal.

Video: Selects the display mode of output video signal.

Setup: Selects the type of setup to be performed, and calls up the appropriate setup menu.

To remove the main menu

Press the MENU button/key again.

To select the type of input video signal

Note

During recording, playback or displaying frozen images (when the CAPTURE button is lit), the type of input video signal cannot be selected.

1 Press the \uparrow or \downarrow button/key to move the cursor to the “Input” line.

```
[Main]
◀Input: RGB
Video: Frame

Setup
Source
Capture/Rec
Play
Disp
System
User
[↑↓] Select [←→] Set
```

Cursor

2 Press the \leftarrow or \rightarrow button/key to change the setting to the desired type.

RGB: RGB signal

S Video: S video signal

Video: Composite video signal

Menu Operations

To select the display mode of output video

Note

The following setting is valid for outputting freeze or playback pictures.

- 1 Press the or button/key to move the cursor to the “Video” line.

```
[Main]
Input: RGB
▶Video: Frame

Setup
Source
Capture/Rec
Play
Disp
System
User
[↑↓] Select [↔] Set
```

- 2 Press the or button/key to change to the desired setting.

Frame: Frame display

Field1/Field2: Field display. Switch the field by setting to Field1 or Field2.

Calling up a setup menu

- 1 Move the cursor to the desired item using the or button/key.

```
[Main]
Input: RGB
Video: Frame

Setup
▶Source
Capture/Rec
Play
Disp
System
User
[↑↓] Select [EXEC] Next Menu
```

The contents of the setup are as follows:

Source: Setup information for the input signal

Capture/Rec: Setup information for the frame memory and disc recording operations

Play: Setup information for playback operations

Disp: Setup information for the operating condition display

System: Setup information regarding the DKR-700/700P system

User: Setup information for setting user data (registration/recall)

- 2 Press the EXEC button (or the EXE key on the RM-C700).

The selected setup menu appears.

Structure and Usage of the Source Setup Menu

Calling up the Source Setup menu

- 1 In the main menu, press the or button/key to move the cursor to the “Source” line.

```
[Main]
Input: RGB
Video: Frame

Setup
▶Source
Capture/Rec
Play
Disp
System
User
[↑↓] Select [EXEC] Next Menu
```

- 2 Press the EXEC button (or the EXE key on the RM-C700).

The Source Setup menu appears.

```
[Source Setup]
▶Adjust R:      0
   G:          0
   B:          0
   Hue:        0      a)
   Color:      0      a)
   Reset

Sync on Green: Off
B/W:           Off
Skew Comp:     Off      a)
H Position:    0
V Position:    0
[↑↓] Select [↔] Set [MENU] Menu
```

a) This item is for the DKR-700 only.

To return to the main menu

Press the MENU button/key.

To change the setup values

- 1 Move the cursor to the desired item using the \uparrow or \downarrow button/key.
- 2 Change the setting using the \leftarrow or \rightarrow button/key.

The settings for each item are as follows:

Adjust R/G/B: Used to adjust the gain for each component of an RGB signal. Values can be set in the range from -127 to +127.

Adjust Hue (DKR-700 only): Used to adjust the hue of a composite or an S video signal. Values can be set in the range from -127 to +127.

Adjust Color (DKR-700 only): Used to adjust the color level of a composite or an S video signal.

Reset: Used to reset the R, G, B, Hue, and Color settings to the values before changing.

Resetting the values

Move the cursor to the “Reset” line using the \uparrow or \downarrow button/key and press the EXEC button (or the EXE key on the RM-C700).

Sync on Green: When using an RGB input signal which has the Sync signal multiplexed with the G signal, set this to On.

Note

The setting of this item automatically becomes On when the “Capture Trig” in the Capture/Rec Setup menu is set to the Camera1 or Camera2.

B/W: Set to On to use a black and white input signal.

Note

When setting this item to On, set the item “Disp Color” in the System Setup menu to White (*see page 42(E)*).

Skew Comp (DKR-700 only): Set to On to use input signals containing jitter.

H Position: Specify the horizontal display position on the screen when this unit is in source mode. (-10 to +10 points)

V Position: Specify the vertical display position on the screen when this unit is in source mode. (-2 to +10 points for DKR-700, -5 to +10 points for DKR-700P)

Structure and Usage of the Capture/Rec Setup Menu

Calling up the Capture/Rec Setup menu

- 1 In the main menu, press the \uparrow or \downarrow button/key to move the cursor to the “Capture/Rec” line.

```
[Main]
Input: RGB
Video: Frame

Setup
Source
▶ Capture/Rec
Play
Disp
System
User
[↵] Select [EXEC] Next Menu
```

- 2 Press the EXEC button (or the EXE key on the RM-C700).

The Capture/Rec Setup menu appears.

```
[Capture/Rec Setup]
▶Quality:      Fine
Rec Mute:     Off
Flush:        0 sec
Capture Trig: FS2
FS2 Mode:     Capture/Rec
Frame/Field: Frame
Live:         EE
Review:       Off
Review Mode:  Play
Auto Rec:     Off
Overwrite:    Off
[↵] Select [↔] Set [MENU] Menu
```

To return to the main menu

Press the MENU button/key.

Menu Operations

To change the setup values

- 1 Move the cursor to the desired item using the \leftarrow or \rightarrow button/key.
- 2 Change the setting using the \uparrow or \downarrow key.

The settings for each item are as follows:

Quality: Used to select the quality of image recording.

Non Comp: Recording uncompressed images

Fine: Recording high-quality images

Normal: Recording normal-quality images

Quick 2: Quick recording normal-quality images (recording the quick access images and search images only)

Quick 1: Quick recording (recording the quick access images and search images only)

Rec Mute: To reduce the recording time, set this to “On”. If this is set to “On”, the monitor is blanked (“muted”) during recording. If this is set to “Off”, the monitor is not blanked (“muted”) during recording.

Flush: Set the time delay after recording an image until recording the image information. The setting values are 0, 13 seconds, 23 seconds, 33 seconds, 1 minute, 3 minutes, 5 minutes, 10 minutes, and unlimited (∞).

If you select a value other than 0, pressing the following buttons or keys makes it possible to start recording the image information.

- EXEC button or EXE key
- PLAY button/key (Playback starts after the image information is recorded.)
- EJECT button (The disc is ejected after the image information is recorded.)
- IMAGE button/key. (The image index appears on the screen after the image information is recorded.)

Capture Trig: Used to select the method by which recording is controlled.

FS2: Use a 1-contact foot switch (connect to the FS2 connector). When this option is selected, also set the next item, “FS2 Mode”.

Camera1: Using a DXC-930 series/950 series (DXC-960/970MD for medical use) Video Camera (RGB/SYNC connection), record in long exposure mode. (The setting of “Sync on Green” in the Source Setup menu automatically becomes On.)

Note

Before selecting Camera1, put the DXC-930 series/950 series (DXC-960/970MD for medical use) Video Camera into slow shutter mode. It is impossible to control the DKR-700/700P with the camera in a normal mode. If Camera1 should be selected during operation with the camera, turn off the camera and change camera settings.

Camera2: Using a DXC-950 series (DXC-970MD for medical use) Video Camera (RGB/SYNC connection), record in the flash synchronized mode. (The setting of “Sync on Green” in the Source Setup menu automatically becomes On.)

Note

When selecting Camera2, set the camera to flash mode.

Ext1/Ext2: Using some other connected external equipment to control the capture trigger.

FS2 Mode: Selects the operation mode when using a 1-contact foot switch.

Capture/Rec: Begin recording an image as soon as it is captured to memory.

Capture: Just capture images to memory.

Alternate: Capture an image to memory with the first operation, and write it to disc on the second operation.

Frame/Field: To capture images in memory and record on the disc in frame units, select “Frame.” To capture images in memory and record on the disc in field units, select “Field.” If using a field-transfer camera with synchronized flash, select “Field.”

1) EE (Electric to Electric) signal: A signal which is input to the recording circuitry, but passes to the outputs having gone through electrical circuits.

Live: Selects whether the input video signal will be output as an “EE” (Electric to Electric)¹⁾ signal, or passed directly to the outputs (as a simple loop-through “Thru” signal).

If this item is set to either EE/Thru or Thru, the next menu item will change from “Review” to “Timer.”

Note

While a menu is displayed on the screen, the output will be an EE signal even when either EE/Thru or Thru is selected. Remove the menu before setting this item.

EE: The output will always be an EE signal.

EE/Thru: The output will be an EE signal except while recording, when the output will switch to a Thru signal.

Thru: The output will be a Thru signal, except for when an image from the DKR-700/700P internal memory is being output (such as when the CAPTURE key is pressed).

Review/Timer: When the previous menu item “Live” is set to EE, this item will be “Review.”

When “Live” is set to EE/Thru or Thru, this item will change to “Timer.”

Review: To automatically review an image immediately after it is recorded, set this item to the desired playback time. The time may be set to 1 second, 3 seconds, 5 seconds, 7 seconds, 10 seconds or Cont (continuous). To turn off the review function, set this item to Off.

Timer: Select whether or not to display a memory image after it is recorded. When set to On, the image stored in memory will be displayed for several seconds, and then the display will revert to the input image. If set to Off, the input image will be displayed immediately. When you set Rec Mute to “On”, Timer automatically changes to “Off”.

Review Mode: Select the image to be reviewed.

When set to “Capture”, the recorded image held in memory is reviewed; when set to “Play”, the image(s) on the disc is (are) reviewed.

Auto Rec: When performing automatic recording, this item sets the interval at which images are to be recorded. The time may be set in a range from 5 seconds to 60 minutes. To turn off the automatic recording function, set this item to Off.

Overwrite: To allow recording over images already recorded on the disc, set to On.

Structure and Usage of the Play Setup Menu

Calling up the Play Setup menu

- 1 In the main menu, press the  or  button/key to move the cursor to the “Play” line.

```
[Main]

Input: RGB
Video: Frame

Setup
Source
Capture/Rec
▶ Play
  Disp
  System
  User
[↑↓] Select [EXEC] Next Menu
```

- 2 Press the EXEC button (or the EXE key on the RM-C700).

The Play Setup menu appears.

```
[Play Setup]

▶Mode:      Auto      a)
Play Mute: Off
Auto Play
Interval:    5 sec
Direction: Forward
Auto Start: Off

[↑↓] Select [↔] Set [MENU] Menu
```

a) This item is set to Main on the DKR-700P at shipping.

To return to the main menu

Press the MENU button/key.

Menu Operations

To change the setup values

- 1 Move the cursor to the desired item using the \uparrow or \downarrow button/key.
- 2 Change the setting using the \leftarrow or \rightarrow button/key.

The settings for each item are as follows:

Mode: Selects the image playback mode.

Auto: Play back the main image after first displaying the quick access image.

Main: Play back the main image.

Quick: Play back the quick access image.

Play Mute: Select whether or not to blank (i.e. “mute”) the screen until a playback image is displayed.

On: The screen is blanked (“muted”). This reduces the time until the image appears.

Off: The screen is not blanked (not “muted”).

Auto Play

Interval: When performing automatic playback, this item sets the interval at which images are to be displayed. The time may be set to any of 3 seconds, 5 seconds, 7 seconds, 10 seconds, 15 seconds, 20 seconds, 30 seconds, 40 seconds, 50 seconds, 1 minute, 2 minutes, 3 minutes, 5 minutes, 7 minutes, and 10 minutes.

Direction: When performing automatic playback, this item sets the order in which images are to be displayed.

Forward: Displays images from smaller to larger image numbers.

Reverse: Displays images from larger to smaller image numbers.

Auto Start: Select whether or not to begin automatic playback when the DKR-700/700P power is turned on.

On: When the power is turned on while an image disc is inserted, automatic playback will begin (in accordance with the settings in the above menu item “Auto Play”).

Off: Automatic playback will not occur.

Structure and Usage of the Disp Setup Menu

Calling up the Disp Setup menu

- 1 In the main menu, press the \uparrow or \downarrow button/key to move the cursor to the “Disp” line.

```
[Main]

Input: RGB
Video: Frame

Setup
Source
Capture/Rec
Play
▶ Disp
System
User
[↵] Select [EXEC] Next Menu
```

- 2 Press the EXEC button (or the EXE key on the RM-C700).

The Disp Setup menu appears.

```
[Disp Setup]

▶ Mode: On
Image No: On
Quality: Off
Remain: Off
Disc Name: Off
Title: Off
ID No: Off
Date: Off
Time: Off

[↵] Select [↵] Set [MENU] Menu
```

To return to the main menu

Press the MENU button/key.

To change the setup values

- 1 Move the cursor to the desired item using the \uparrow or \downarrow button/key.
- 2 Change the setting using the \leftarrow or \rightarrow button/key.

The settings for each item are as follows:

Mode: To display the input signal type and the DKR-700/700P's operating condition, set to On.

Image No: To display the image number of the next image to be recorded or of the image currently being played back, set to On.

Quality: To display the image quality setting, for either record or playback, set to On.

Remain: To display the number of images that may be recorded in the remaining disc space, set to On.

Disc Name: To display the disc name of the disc currently in the DKR-700/700P, set to On.

Title: To display the name of the next image to be recorded or of the image currently being played back, set to On.

ID No: To display the ID number of the next image to be recorded or of the image currently being played back, set to On.

Date: To display the recording date (y/m/d) of the image currently being played back, set to On.

Time: To display the recording time (h/m/s) of the image currently being played back, set to On.

Structure and Usage of the System Setup Menu

Calling up the System Setup menu

- 1 In the main menu, press the \uparrow or \downarrow button/key to move the cursor to the "System" line.

```
[Main]
Input: RGB
Video: Frame

Setup
Source
Capture/Rec
Play
Disp
▶ System
User
[↑↓] Select [EXEC] Next Menu
```

- 2 Press the EXEC button (or the EXE key on the RM-C700).

The System Setup menu appears.

```
[System Setup]
▶Flash: Off
Rec Tally: High
RS-232C: RM-C700
Baud Rate: 9600
FSI Mode: A
Disp Color: White
Beep: On
Key Click: Off
Rec End: Off
Date: 96-01-01
Time: 00:00
[↑↓] Select [↔] Set [MENU] Menu
```

To return to the main menu

Press the MENU button/key.

Menu Operations

To change the setup values

- 1 Move the cursor to the desired item using the \uparrow or \downarrow button/key.
- 2 Change the setting using the \leftarrow or \rightarrow button/key.

The settings for each item are as follows:

Flash: To generate a trigger signal from the FLASH connector, set this item to On.

Rec Tally: Select the polarity of the Rec Tally signal output from the FS2 connector. You can select “High” or “Low”.

RS-232C: Select the device to be connected to the RS-232C connector. (This item can be set only by front panel operation.)

RM-C700: The RM-C700 Remote Control Unit.

Ext CPU: A computer. If this choice is selected, the next item (“Baud Rate”) must also be set.

Baud Rate: When controlling the DKR-700/700P from a computer, the baud rate must be set to match that of the computer’s serial port. Allowable values are 1200, 2400, 4800, and 9600.

FS1 Mode: Selects the operation mode when using an optional FS-30 Foot Switch. (See page 25(E).)

Disp Color: Used to set the colors of the display characters. Available choices are white, red, green, yellow, blue, magenta, and cyan.

Beep: To cause the DKR-700/700P to beep upon errors and the completion of operations, set this to On. When set to On, the next items, “Key Click” and “Rec End”, may also be set.

Key Click: Setting this to On will cause the DKR-700/700P to emit a short click whenever one of the keys on the front panel or on the RM-C700 is pressed.


Rec End: Set this to “On” if you wish to sound a beeper when recording is completed.

Date: The date may be set into the internal clock.


Time: The time may be set into the internal clock.

Setting date


- 1 Move the cursor (\blacktriangleright) to the “Date” line using the \uparrow or \downarrow button/key and press the \leftarrow or \rightarrow button/key to display the cursor ($_$).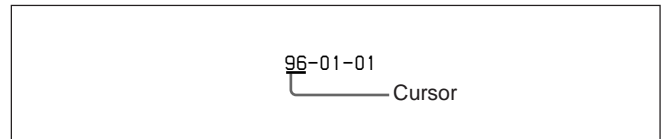

- 2 Move the cursor ($_$) to the item (day, month, or year) to be set using the \leftarrow or \rightarrow button/key.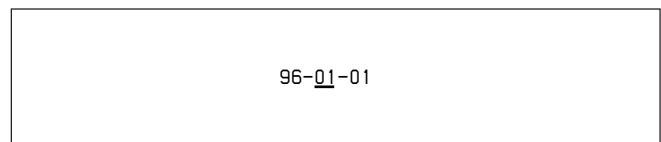

- 3 Using the \uparrow or \downarrow button/key, change the value of the selected item.

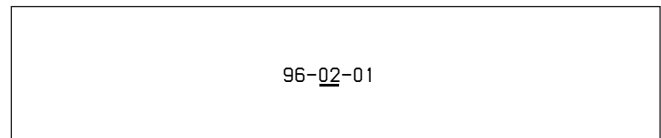- 4 To change other items, repeat steps 2 and 3. When the date is set correctly, go on to step 5.


- 5 Press the EXEC button (or the EXE key on the RM-C700).

The cursor ($_$) disappears.


Setting time

- 1 Move the cursor (▶) to the “Time” line using the  or  button/key and press the  or  button/key.

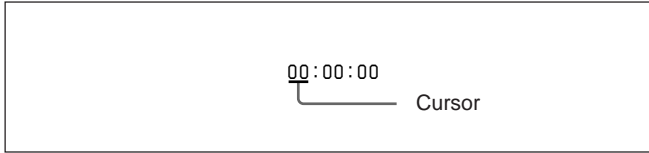The cursor (▬) and “second” indication appear.

- 2 Move the cursor (▬) to the item (second, minute, or hour) to be set using the  or  button/key.

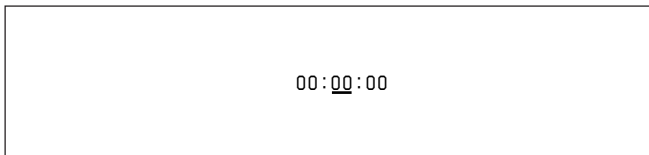- 3 Using the  or  button/key, change the value of the selected item.

- 4 To change other items, repeat steps 2 and 3. When the time is set correctly, go on to step 5.

- 5 Press the EXEC button (or the EXE key on the RM-C700).

The cursor (▬) and “second” display disappear.

Structure and Usage of the User Setup Menu

Calling up the User Setup menu

- 1 In the main menu, press the  or  button/key to move the cursor to the “User” line.

```
[Main]

Input: RGB
Video: Frame

Setup
Source
Capture/Rec
Play
Disp
System
▶ User
[↑↓] Select [↔] Set
```

- 2 Press the EXEC button (or the EXE key on the RM-C700). The User Setup menu appears.

```
[User Setup]

▶ Load: 1
Save: 1

[↑↓ ↔] Select [EXEC] Execute
```

To return to the main menu

Press the MENU button/key.

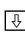
Menu Operations

Using the User Setup menu

The User Setup menu allows the settings for all menus to be registered as “user data,” and recalled as needed.

Registering user data

Up to three different sets of user data may be created, each identified with a number.

- 1 Set all menu items as desired, and then call up the User Setup menu.
- 2 Press the  or  button/key to move the cursor to the “Save” line.

```
[User Setup]
Load: 1
▶Save: 1

[↑ ↓] Select [EXEC] Execute
```

- 3 Press the  or  to select 1, 2 or 3 as the register number of the current menu settings.

A number highlighted in reverse video is already used.

```
[User Setup]
Load: 1
▶Save: 2

[↑ ↓] Select [EXEC] Execute
```

- 4 Press the EXEC button (or the EXE key on the RM-C700).

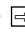
The current menu settings are registered as user data.

Recalling user data

- 1 Call up the User Setup menu.
- 2 Press the  or  button/key to move the cursor to the “Load” line.

```
[User Setup]
▶Load: 1
Save: 1

[↑ ↓] Select [EXEC] Execute
```

- 3 Press the  or  to select the number of the desired user data.

The data whose number is highlighted in reverse video can be recalled.

```
[User Setup]
▶Load: 2
Save: 1

[↑ ↓] Select [EXEC] Execute
```

- 4 Press the EXEC button (or the EXE key on the RM-C700).

The current menu settings are replaced with the recalled user data, and the DKR-700/700P is in source mode.

Specifications

General

Classification of equipment

- Type of protection against electric shock: Class I equipment
- Degree of protection against electric shock: Type B equipment (DKR-700 only)
- Degree of protection against harmful ingress of water: Ordinary equipment
- Degree of safety of application in the presence of a flammable anaesthetic mixture: Not protected equipment
- Mode of operation: Continuous operation

Power requirements

DKR-700: 120 V AC,
50/60 Hz
DKR-700P: 220/230/240 V AC,
selectable, 50/60 Hz

Power consumption

DKR-700: Max. 0.6 A
DKR-700P: Max. 0.3 A

Operating temperature

5°C to 35°C (41°F to 95°F)

Operating humidity

20% to 80% (no condensation)

Transport and storage temperature

-20°C to +60°C (-4°F to +140°F)

Transport and storage humidity

20% to 90% (no condensation)

Mass

Approx. 7 kg (15 lb 6 oz)

External dimension

212 × 130 × 380 mm (8 ³/₈ × 5 ¹/₈ ×
15 inches) (w/h/d)

Laser diode properties

Material	Ga Al As
Wave length	780 ±20 nm
Emission duration	Continuous
Laser output power	35 mW (max.)
Beam divergence	(H) typ. 12°, min. 10°, max. 18.5° (V) typ. 24°, min. 20°, max. 28.5°

Recording/playback system

Recording medium

Data MiniDisc

Recording format

Picture MD

Compression/decompression format

JPEG baseline (fixed length)

Image size Main image: 656 pixels × 496 lines
(DKR-700), 768 pixels × 592
lines (DKR-700P)

Quick access image: 640 pixels ×
480 lines

Index image: 80 pixels × 60 lines

Sampling (for main image)

Compressed: Y:U:V=4:2:2

No compressed: R:G:B=4:4:4

Video characteristics

Signal format

DKR-700: NTSC, 525 lines, 60
fields/second interlaced

DKR-700P: PAL, 625 lines, 50
fields/second interlaced

Quantizing

R, G, B: 8-bit

Sampling rate

DKR-700: 12.273 MHz

DKR-700P: 14.750 MHz

Horizontal resolution

DKR-700: Min. 480 TV lines

DKR-700P: Min. 500 TV lines

Input/output connectors

Video input/output connectors

VIDEO

BNC-type × 1

1.0 Vp-p ±10%, 75 Ω, sync
negative

S-VIDEO

Mini-DIN 4-pin × 1

Y: 1.0 Vp-p ±10%, 75 Ω, sync
negative

C: 0.286 Vp-p ±10% (DKR-700),
0.3 Vp-p ±10% (DKR-700P),
75 Ω

RGB/SYNC

BNC-type × 4

R, G, B: 0.7 Vp-p ±10%, 75 Ω

SYNC: 4.0 Vp-p ±10%, 75 Ω

Remote connectors

RS-232C

D-sub 9-pin × 1

Receiving levels (at 5-kilohm load)

High: +5 V to +15 V

Low: -5 V to -15 V

Appendixes

	Transmitting levels (at 3-kilohm load) High: typ. +7.5 V Low: typ. -7.5 V
FS1	D-sub 9-pin ×1 (for an optional FS-30 Foot Switch connection)
FS2	Stereo minijack ×1 (for a 1-contact foot switch connection) 39 kilohm pull-up (5 V DC)
EX-CTL	Minijack ×1 (for trigger output to external equipment) Open collector output (max. 12 V DC, 0.1 A)
FLASH	Minijack ×1 (for trigger output to an optional flash unit) Output: 200 V DC (allowable voltage), 0.3 A
SCSI	50-pin, half-pitch×1 Output levels High: 2.5 V to 5.25 V Low: 0 V to 0.4 V Input levels High: 2.0 V to 5.25 V Low: 0 V to 0.8 V

Accessories supplied

MMD-140 MD Data Disc (3)
AC Power Cord (1)
DKR-700 Plug-In Software (2)
Instructions for Use (1)

Accessories not supplied

RM-C700 Remote Control Unit
FS-30 Foot Switch
SMF-3036C RS-232C Cable
KB-SPC06 SCSI Cable (SANWA supply)

Recommended equipment

DXC-930 series/950 series (DXC-960/970MD for medical use) Color Video Camera
UP-1000 series/5500 series Color Video Printer
PVM series Color Video Monitor

Design and specifications are subject to change without notice.

Error Messages and Warning Messages

If an error occurs while the unit is operating, or if you make an inappropriate operation, an error message or warning message appears on the video monitor. The following table shows the meaning of each message, and the appropriate action to take.

Error messages

Message	Meaning	Action to take
Rec Error	There is a fault in the MD drive, and as a result a verify error occurred in the management information file during recording; the disc is ejected.	Contact your Sony dealer.
Loading Error	There is a fault in the MD drive, and as a result a verify error occurred when a disc was inserted or when the unit was powered on with a disc inserted; the disc is ejected.	Contact your Sony dealer.
File Read Error	When the currently inserted disc was last being accessed, the unit was powered off. As a result, the size check on the management information file now produces an error.	Stop using the currently inserted disc.
Init/Erase Error	There is a fault in the MD drive, and as a result an Initialize/All erase operation failed, or after an Initialize/All erase operation, a verify error occurred; the disc is ejected.	Contact your Sony dealer.
File Form Error	A disc of an incompatible PICTURE MD format with a higher version number was inserted. Alternatively, after such a disc was inserted, an attempt was made to access it.	Stop using the currently inserted disc.
TOC Read Error	There is a fault in the MD drive, or the disc is corrupted, and as a result the TOC (Table of Contents) of the current disc cannot be read.	Try another disc. If that also produces the same error, contact your Sony dealer.

Appendixes

Warning messages

Message	Meaning	Actin to take
No Disc	An attempt was made to record or play back, with no disc inserted.	Insert a disc.
Not Initialized	An unformatted disc was inserted, or an attempt was made to access it.	Press the ALL ERASE/INIT key to format the disc.
Write Protect	With a write-protected disc inserted, an attempt was made to record or delete.	Remove the write protection from the disc.
No Image	An attempt was made to play back or search, but the inserted disc has no images recorded on it.	Insert a disc on which images are recorded.
No Data	An attempt was made to display the image information list, but the inserted disc has no image information recorded on it.	Insert a disc on which the image information is recorded.
No More Rec	An attempt was made to record, but the inserted disc has no remaining recording capacity.	Make sure the disc has sufficient recording capacity before recording.
Batt Empty	The unit was powered on with the lithium battery exhausted.	Contact your Sony dealer.
No Info. Area	The inserted disc has been recorded on a different system, and there is no space to write the disc name, file names, ID numbers, image names, and other items; an attempt was made to enter or amend this information.	Do not enter or amend the information.
Erase Inhibit	An attempt was made to erase or overwrite an image which was written on a different type of unit, and which has erase-inhibit set.	Do not erase or overwrite the image.
Data Not Save	When executing Load in the User Setup menu, you specified a data number for which the data has not been entered.	Specify a data number for which the data has been entered.
Not Source Mode	You pressed the All Erase key with the unit not in the source mode.	Press the Source key to put the unit in the source mode before pressing the All Erase key.
Not Play Mode	You pressed the Erase key with the unit not in the play mode.	Press the Play key to put the unit in the play mode before pressing the Erase key.
Audio Disc	The disc is ejected, because it is an audio MiniDisc.	Insert a data MiniDisc.
Not MD Data Disc	The disc is ejected, because it is neither a data MiniDisc nor an audio MiniDisc.	Insert a data MiniDisc.

Troubleshooting

Should a problem occur with the operation of the DKR-700/700P, check the following list before contacting a Sony representative, and see if the remedies given solve the problem.

Symptom	Cause/Remedy
The POWER switch is turned on, but the DKR-700/700P does not power up.	Check to see that the power source is properly connected (page 16(E)).
The REC button is pressed but recording does not occur.	<ul style="list-style-type: none"> • The disc may be write-protected. Remove the write protection (page 18(E)). • The disc may be full (and the menu set to disallow overwriting). Use a new, initialized disc (page 19(E)), or change the Capture/Rec Setup menu setting to allow overwriting (page 41(E)). • The disc may not be initialized to the proper format. Initialize the disc (page 19(E)).
The upper portion of the screen oscillates horizontally.	<ul style="list-style-type: none"> • The input signal may be non-interlaced. Use an interlaced input signal. • When using the DKR-700, the input signal may have a horizontal frequency of 15.75 MHz. Set the "Skew Comp" setting to Off in the Source Setup menu (page 40(E)).
The DKR-700/700P is in still mode, and does not recognize input when connected to the DXC-930 series/950 series (DXC-960/970MD for medical use) Video Camera.	The menu may be set to "Camera1", but the camera is not in slow shutter mode. Stop sync input from the camera, or power off the camera and change the Capture/Rec Setup menu setting (page 41(E)) or camera settings.


Timing for Capturing an Image Using a Foot Switch

If you capture an image using a foot switch, use the DIP switch on the rear panel of the DKR-700/700P to set capture timing. (See page 17(E).)

The following figures show which frame or field is captured on the selected timing.


Capturing images in frame units

Select “Frame” for “Frame/Field” in the Capture/Rec Setup menu. (See page 41(E).)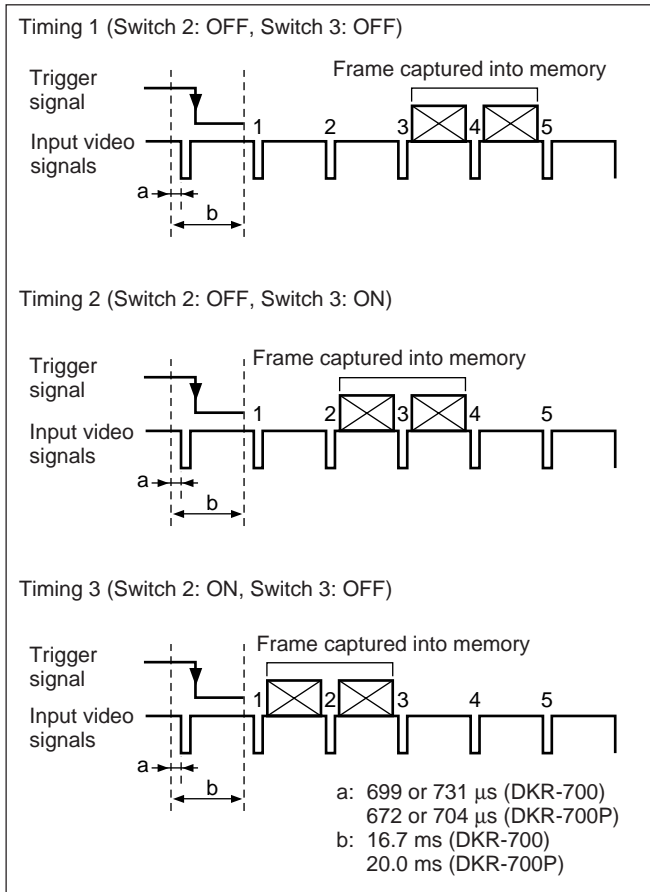

Capturing images in field units

Select “Field” for “Frame/Field” in the Capture/Rec Setup menu. (See page 41(E).)

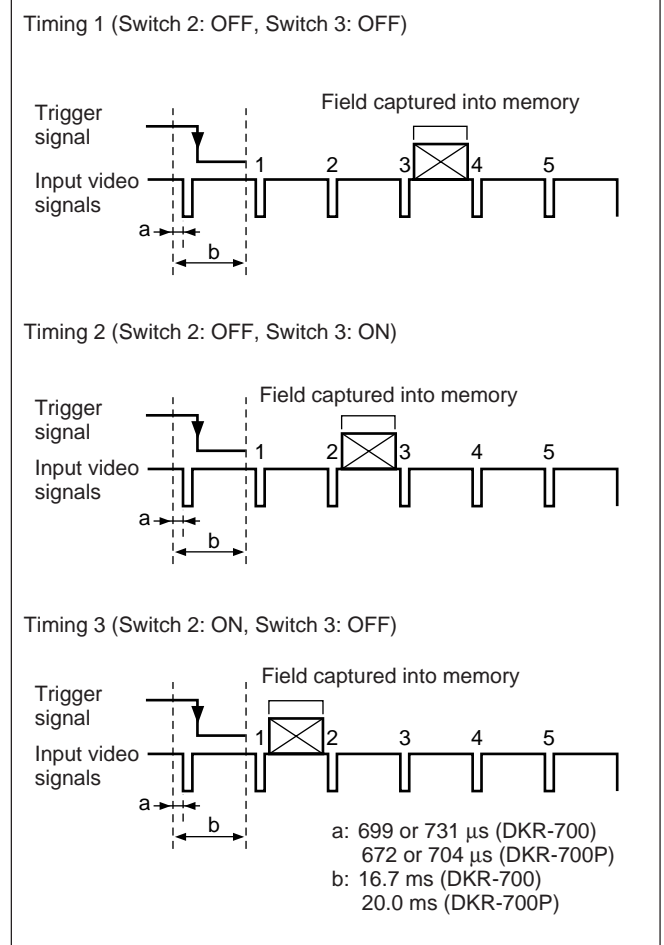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